

3) Connecting both the patient's Call Cord and the Fall Savers Sentry Deluxe monitor (91600) to a single connection point on your nurse call system.

If you only have one available connection point on your nurse call system to which you need to connect both the patient's call cord and the Fall Savers Sentry Deluxe monitor then you will need to use the T-Connector (0440-157) supplied with your Fall Savers Nurse Call Connection Kit (NCKIT). First connect your Fall Savers Sentry Deluxe monitor (91600) to your nurse call system using the Fall Savers Nurse Call Connection Kit (NCKIT) as explained above for either a standard mono maker or standard stereo maker nurse call system.


Unplug the NCMONO nurse call cable from the Fall Savers Sentry Deluxe monitor (91600) and plug the T-Connector (0440-157) directly into receptacle on the base of the Fall Savers Sentry Deluxe monitor (91600) marked "Nurse Call". Push down firmly to ensure that the T-Connector is secure in the base of the monitor. Plug the NCMONO nurse call cable into one receptacle on the T-Connector and plug the patient's call cord into the other receptacle on the T-Connector. You should have both the NCMONO nurse call cable and the patient's call cord plugged into the T-Connector and the T-Connector plugged into the receptacle on the base of the monitor marked "Nurse Call". Test the system to ensure that the nurse call system activates both when the Fall Savers Sentry Deluxe monitor (91600) goes into alarm and when the patient's call cord button is pressed.

4) Testing the System


Once you have connected the Fall Savers Sentry Deluxe Monitor to your nurse call system you must TEST to ensure that a call is activated on your nurse call system when the monitor goes into alarm. To conduct this test plug the sensor pad into the receptacle at the base of the Fall Savers Sentry Deluxe Monitor marked "Sentry Mat". With the sensor pad connected to the monitor press firmly with your palm onto the centre of the sensor pad maintaining pressure for five seconds. When you apply pressure to the centre of the sensor pad the monitor should beep once to indicate that the monitor has detected weight on the sensor pad. After five seconds lift your palm off the sensor pad. The monitor should then go into alarm and a call should be activated on your nurse call system. If you system does test properly then do not use and call Fall Savers on 0800 032 4789.

5) Connecting to Nurse Call Systems other than standard Mono Maker or Stereo Maker types/


The Fall Savers Nurse Call Connection Kit (NCKIT) can be used only for connecting to standard type mono or stereo nurse call systems. The components used in this Fall Savers Nurse Call Connection Package will not work with other types or nurse call systems or with "breaker type" mono and stereo nurse call systems. If your nurse call system does not activate when the Fall Savers Sentry Deluxe Monitor goes into alarm or if you can not cancel the call on your nurse call system when the monitor is connected to your nurse call system then please call Fall Savers on UK Freephone 0800 032 4789.

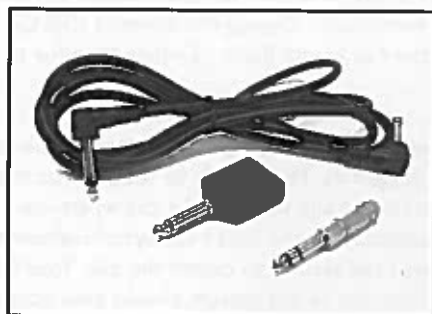


NCKIT

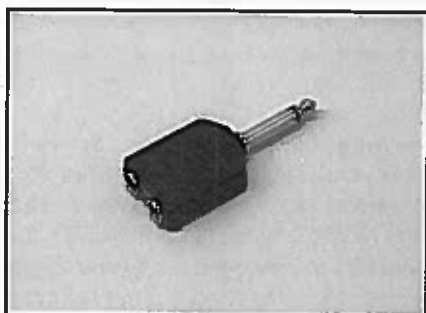


0440-137

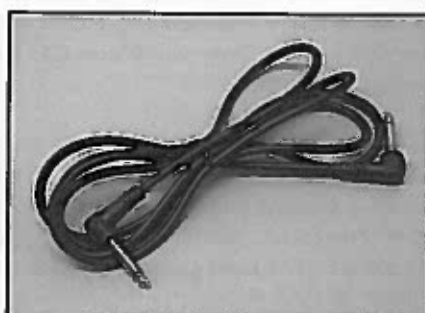

NCMONO

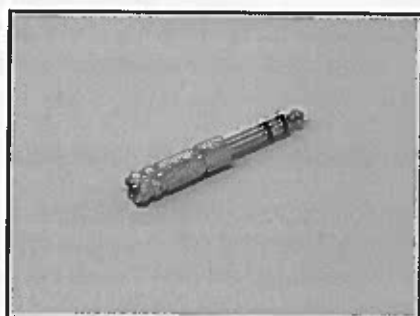

NCPLUGSTEREO

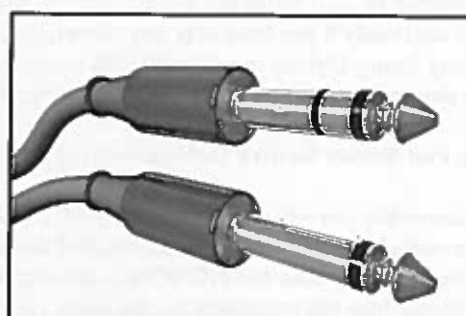

**MONO AND
STEREO MAKER**