

## User Instruction for Foot Stool with Safety Handrail

**Please read and retain these instructions, they provide essential information for correct and safe use. These instructions are guidelines only.**

### INTRODUCTION

The foot stool with safety handrail is a simple and sturdy stool consisting of a chrome plated handrail and a chrome plated steel frame with a non-skid pad for placing the feet. The handrail provides additional support and safety.

### SPECIFICATION

Overall Size	Weight	Material	Max. Load Weight
470 x 350 x 870 mm	4.1 kgs	Steel+PVC(pad)+Rubber(tips)	115 kgs

### INSTRUCTIONS FOR ASSEMBLY AND USE

1. Align the U-shaped seat legs across each other and tighten the screws through the centre point of the tube.
2. Place the seat legs upside down on the flat table with the underside facing upwards. Align the U-shaped seat legs with the underside of the table (following the indentations moulded into the underside of the table).
3. Feed the four screws through the centre point of each tube to fix the U-shaped legs in position and tighten screw slightly.
4. Attach the handrail to the seat legs and tighten the screws slightly.
5. Ensure that all four legs of the foot stool are in contact with the floor.

### WARNING

- Users with limited physical capabilities should be supervised or assisted when using the foot stool.
- Test for stability before use.
- Ensure your weight is evenly distributed when standing on the foot stool. Improper or uneven loading may result in the stool tipping over.
- Do not tip or put an offset load onto the foot stool.
- Do not exceed the max. load weight of 115 kgs.
- Periodically inspect for wear and damage.
- If any fault occurs, please contact the supplier from whom you purchased it as soon as possible.

### MAINTENANCE

The foot stool with safety handrail is designed for minimum maintenance. Clean it with a wet cloth.