

F19739	Adjustable Height Stool	F19741	Adjustable Height Stool with Armrests
F19893	Adjustable Height Stool with Armrests & Backrest	L33236	Adjustable Height Stool with Armrests & Padded Backrest
F21541	Adjustable Height Stool with Padded Armrests & Padded Backrest	L63344	Extra Low Adjustable Height Stool with Armrests & Padded Backrest
L63356	Extra Low Adjustable Height Stool with Padded Armrests & Backrest		

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

The perching stool range is intended to be used to provide a stable, seated support, where a user is unable to stand for long periods, the seat slopes forward to allow the user to achieve a relaxed “perching” position when sitting and also to assist when rising.



L63344 Shown

2. SAFETY

Warning: The maximum user weight must not be more than 160Kg (25st).

Do not exceed the maximum user weight limit.

Inspect product at regular intervals for signs of wear or damage

Do not use if you see or suspect a fault.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. HEIGHT ADJUSTMENT

To adjust the stool to the desired height, remove the plastic clips connecting the stools’ legs and leg extensions.

Slide the leg extensions to the required position, ensuring that all legs are equal in height.

Replace the plastic clips ensuring that each clip is fully connected.

Note: Ensure the floor surface is level and even to avoid rocking.

5. MAINTENANCE

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules

Check that all fixing clips are inserted correctly and there is no excessive movement.

Check for general wear or damage.

If you see or suspect a fault do not use – contact your dealer immediately

6. CLEANING INSTRUCTIONS.

Clean regularly, use a general purpose, neutral detergent and warm water

Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.

Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water

Dry thoroughly before use.

7. PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal. This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship.

This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk