


User Instructions

L97779 - Handling Sling

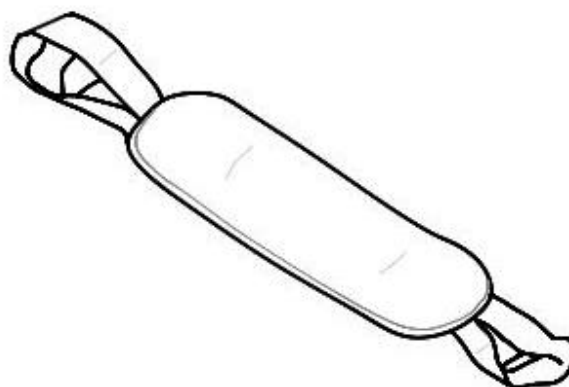

General Safety Warning

Please ensure these instructions are fully read and understood before use. Retain in a safe place for future reference.

Intended Use

The NRS Handling Sling has been designed to extend the carer's reach during a variety of moving and handling procedures – It is particularly useful to assist a person to move to the edge of a chair before standing, also for repositioning in bed where support around the torso is required. The soft lining is gentle on damaged and delicate skin whilst the dual handles provide flexibility and carer comfort.

Leg positioning.

By placing the sling under the leg and holding the handles it is possible for a carer to safely reposition a person's legs in bed.

Important:

The NRS Handling Sling should not be used for lifting a person, or their legs, it is designed only as a repositioning and support aid.

Specification

Main body fabric and fleece: 100% polyester
 Webbing: 100% polypropylene
 Foam Inserts: 100% polyethylene

Dimensions:

Padding: 610mm x 220mm (24 x 8½")

Care and Maintenance

Inspection should be carried out thoroughly and at regular intervals. If the product is damaged or the fabric and materials are in poor condition do not use.

Washing Instructions

Machine wash warm (up to 40°C)

Do not use bleach or biological detergents.

Do not spot clean with strong chemicals

Do not autoclave.

Do not dry clean

Tumble dry Warm. Reduced spin, or air dry at very low temperatures



IMPORTANT

For information on how to obtain the User Instructions in other languages and formats for use by people with visual, reading or cognitive impairments, please contact: info@nrs-uk.co.uk or telephone 0845 120 4522.

Additional copies of this user instruction, together with details of our wide range of aids for daily living, are available on our website:

www.nrs-uk.co.uk.

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB.

Tel: 0845 120 4522 Fax: 0845 121 8112

Email: customerservice@nrs-uk.co.uk

These products are part of a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range please visit our website:

www.nrs-uk.co.uk

or request a catalogue using the number above