



INSTRUCTIONS FOR USE

Max.user weight:



Secure Turn Patient Turner Catalogue Number: L84438

INTENDED PURPOSE:

The Secure Turn Patient Turner has been designed to rotate a standing patient during a transfer from e.g. wheelchair to a chair.

PRECAUTIONS:

Ensure these instructions are fully read and understood before use and please keep in a safe place for future reference.

Please note the patient must not be transported using the device.

Do not exceed the maximum user weight limit of 30st (190Kg).

Ensure the handle is facing the wheel and that the brake is locked before manoeuvring the Secure Turn.

Never assist a patient to alight from the wheeled side of the Secure Turn.

Transfers should only be carried out to the left or right through 45° or 90°.

Patient alighting should only be performed when the brake is engaged and the Secure Turn is stabilised by a carer.

The Secure Turn should be inspected on a regular basis for signs of damage or wear that might affect safe use.

Do not clean this device using an industrial cleaner or high-pressure hose as this may affect the smooth operation of the turntable.

INSTRUCTIONS:

1. Familiarise yourself with the operation of the Secure Turn before attempting to transfer a patient, in particular the brake locking mechanism
2. To manoeuvre the Secure Turn into position, with the handle facing the wheels and brake locked in position, tilt the device back onto the wheels and move into position.
3. Place the seat/chair that the patient is transferring to into position at 45° or 90° to the patient's current position.
4. Place the turntable up to the patient's feet with the wheels facing the carer.
5. The patient should position their feet, evenly spaced, on the turntable mat. The patient may need to move to the edge of the seat before rising to reduce the effort required.
6. With the Carer stabilising the Secure Turn, the patient raises themselves into a standing position.
7. When the patient is standing and stable (holding on to one of the pairs of handles on the frame), the carer releases the brake and rotates the turntable to the required position allowing the brake to lock into position.
8. The carer stabilises the Secure Turn whilst the patient lowers themselves onto the seat/chair.
9. After the patient has removed their feet from the turntable, manoeuvre the Secure Turn away from the patient.



Secure Turn Patient Turner L84438

Instructions For Use



Care & Cleaning

Wipe clean with a soft cloth, moistened with disinfectant solution.

Manufactured in the UK for

Nottingham Rehab Supplies

Clinitron House
Excelsior Road
Ashby Park
Ashby de la Zouch
Leicestershire
LE65 1JG

Tel: 0845 120 4522

Fax: 0845 121 8112

Email: customerservice@nrs-uk.co.uk

For further details of Nottingham Rehab Supplies' extensive range of equipment and aids for daily living please visit our website
www.nrs-uk.co.uk

