

# USER INSTRUCTIONS

## MOWBRAY LITE FLAT PACK

### TOILET FRAME AND SEAT

M66613

Mowbray Lite Flat Pack - Height & Width Adjustable

#### General Safety Warning

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



## 1. INTENDED USE

To assist those who have difficulty in lowering onto or rising from a standard toilet. The comfortable plastic moulded seat offers support in a natural position. An optional floor fixing kit is also available (F11572).

## 2. SAFETY

**WARNING:** The maximum user weight must not be more than 190 kg (30 st).

**DO NOT:** Attempt to stand or climb on the product.

**MAKE SURE:** All legs are set at equal height to avoid rocking.

## 3. UNPACKING


Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### 3.1 Contents:

1. Toilet Seat x 1
2. RH Frame Assembly x 1
3. LH Frame Assembly x 1
4. Crossbars x 2
5. Leg Extensions x 4
6. E-clips x 8

## 4. ASSEMBLY


### 4.1 Fitting the Crossbars

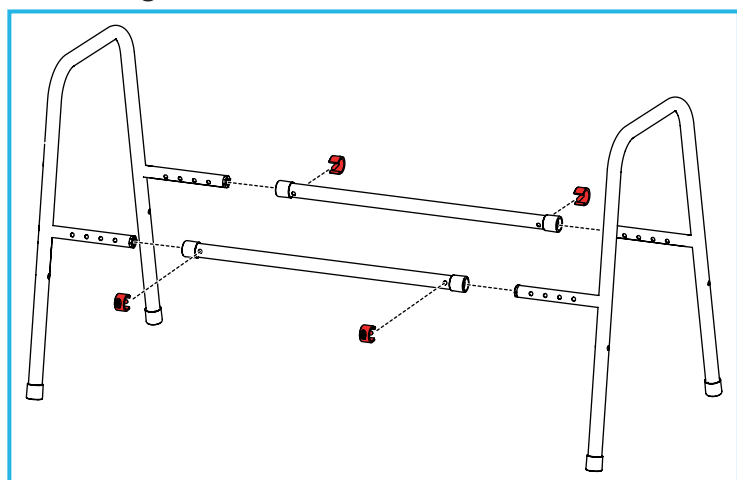

1. Slot the 2 x crossbars onto the side-frame assemblies and adjust to the correct width for the toilet and the user.
2. Align the holes and secure with the 4 x E-clips.

**Note:** Make sure that the side-frames and crossbars have been set to the same width/adjustment holes.


### 4.2 Fitting the Leg Extensions

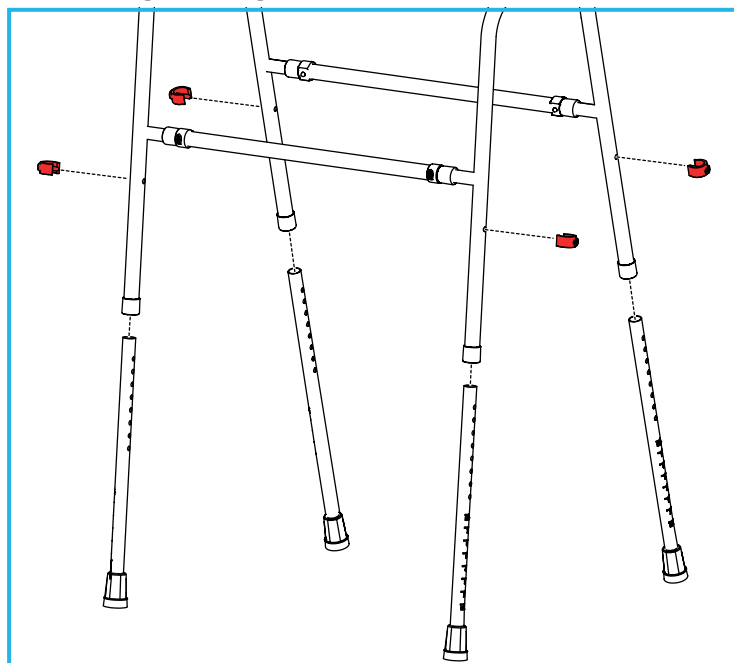

3. Slot the 4 x leg extensions into the side-frame assemblies and adjust to the correct height for the toilet and the user.
4. Align the holes and secure with the 4 x E-clips.

**Note:** Make sure the leg extensions have all been set to the same height.

### 4.3 Fitting the Seat

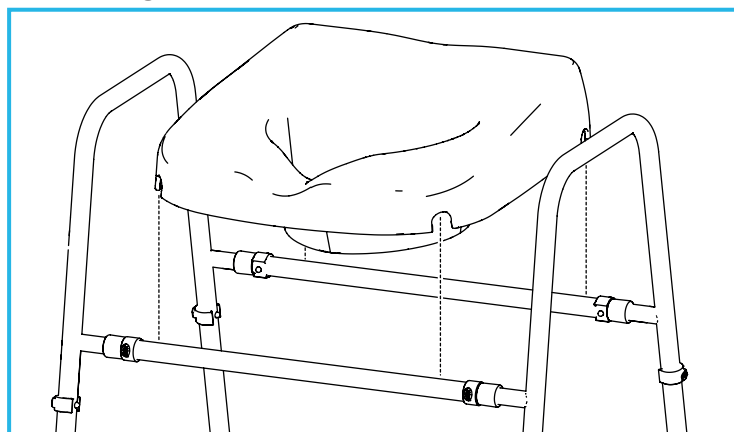

5. Remove the seat from the bag and snap into place on the centre bars of the frame.

## 5. INSTALLATION

1. Lift the toilet seat and lid.
2. Put the Mowbray seat over the toilet with the cutaway of the seat at the front and the integral splash guard placed centrally over the toilet bowl.

**Note:** Make sure the seat and toilet frame are fully secure before use.

## 6. IN USE

### 6.1 Adjust the Height

1. Remove the 4 x leg extension E-clips.
2. Slide the leg extensions to the required height, align the holes and replace the E-clips.

**Note:** Make sure the leg extensions have all been set to the same height.

**Note:** Make sure the seat and toilet frame are fully secure before use.

### 6.2 Adjust the Width

1. Remove the 4 x crossbar E-clips.
2. Slide the side frame assemblies to the required width, align the holes and replace the E-clips.

**Note:** Make sure the side frame assemblies have been set to the same width.

**Note:** Make sure the seat and toilet frame are fully secure before use.

### 6.3 Fitting the Optional Floor Fixing Kit F11572

Please refer to the user instructions provided with the optional floor fixing kit.

## 7. MAINTENANCE

Check that all fixing clips are inserted correctly and that there is no excessive movement.

If a floor fixing kit has been fitted, check the fixing screws are secure.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

## 8. CLEANING INSTRUCTIONS

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Regularly clean with a non-abrasive cleaner.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

This product is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 70°C.

Allow product to fully dry prior to use/storage.

## 9. ACCESSORIES & SPARE PARTS

F11572 - Floor Fixing Kit

F17938 - Ferrule

M66820 - Seat

N43218 - E-clips x 4

For further information please visit our website or call Customer Services on the details below.

## 10. PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 11. PRODUCT GUARANTEE












NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

## 12. GLOSSARY

	Manufacturer		Batch Reference Code		Consult the instructions for use
	Important Note		Authorised representative in the European Community		Medical Device
	Item/Catalogue number		European Conformity / Conformité Européene		UK conformity assessed marking
	Maximum User Weight		Warning		



NRS HEALTHCARE Sherwood House,  
Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire,  
LE67 1UB, UK.



NRS HEALTHCARE 4C Altona Road,  
Blaris Industrial Estate, Lisburn, Co Antrim,  
Northern Ireland BT27 5QB

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)