

USER INSTRUCTIONS

CORNER SHOWER STOOL

H84059

Corner Shower Stool

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

The Corner Shower Stool is designed to sit in the corner of a shower, taking up minimal space, and is height adjustable so that it can be set at the correct height to assist standing.



2. SAFETY

Warning: The maximum user weight must not be more than 127Kg (30st).

Inspect product at regular intervals for signs of wear or damage.

Do not use if you see or suspect a fault.

Do not place weight on the seat when removed from the frame. It is designed to be supported by the frame, and only in this position will it conform to the maximum weight limit stated.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. ASSEMBLY/INSTALLATION

Check the shower tray can take the recommended weight before use.

Adjust the legs using the “E” clips to the required height for comfortable standing.

When adjusting the legs, ensure there is no gap between the “E” clip and the frame.

Ensure that all the legs are fixed in the same position to prevent rocking.

5. MAINTENANCE

Periodically check the legs for excess movement and rubber feet and seat for damage. Do not use if damaged.

6. CLEANING INSTRUCTIONS.

Clean regularly, use a general purpose, mild detergent and warm water.

Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

To disinfect, wipe with a mild chloride

disinfectant (solution of 1000 parts per million). Autoclave at 80°C. Ensure fully dried after cleaning.

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

Household Waste and Recycling Centres.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk