

PRODUCT INSTRUCTION SHEET

REFERENCE: Rutland Trolley with Push Bar '275PB'

DESCRIPTION

- All welded, Duradip coated steel frame with angled sides to provide safe handgrips.
- Injection moulded plastic shelves with 4 raised edges.
- Bottom shelf set forward for ease of walking/dining.
- 4"x 4" (100mm) castors.
- Adjustable in height
- Fixed welded push bar

TECHINICAL SPECIFICATIONS

Shelf size: 16" x 12.5"

Width 45cm (17.5")

Depth 50cm (19.5")

Overall Height Min 88cm (34.5")

Overall Height Max 103cm (40.5")

Push Bar height: 4" from lowest part of handle

Weight 10kg (22lb)

Weight Capacity: 177kg (28st)



USAGE REQUIREMENTS

1. Check all castors are securely fitted before and after use. The nylon bush on the top of the castor must be pushed fully into the extension leg, with no gap visible.
2. When adjusting the legs ensure that there are no gaps between horseshoe clip and frame. The clip spindle must be fully inserted right through the outside frame.
3. Ensure all legs are the same height.
4. The trolley is **not** designed to be sat upon.
5. Although this item is used when walking – IT SHOULD NOT BE USED AS A WALKING AID. All castors must be in contact with the floor at all times during use
6. It is recommended that the trolley is used on level surfaces only.
7. The trays are designed to be a clip on/off fit, for cleaning purposes. To fit the tray, push firmly down and to remove, push upwards from the underside. Make sure trays are level with the framework
8. Clean the frame and tray with mild detergent/cream cleaner and warm water. Allow to dry thoroughly.
9. Do not exceed maximum user weight of 177kg (28 Stones)