OLD CODE	NEW CODE
AA8389	09 111 7811

The Buckingham Walking Frame Caddy

Parts included:
Pannier, Tray,
Mug Holder,
Mug Cover,
Clips x 2

Intended uses

The Buckingham Walking Frame Caddy consists of a tray and pannier, which allows you to safely carry personal items and meals easily and with confidence. Simple to attach to virtually all wheeled walking frames it is light but sturdy, easy to clean and will not upset the balance of the frame.

Instructions for attaching the caddy

This product has been designed for use with non-folding, wheeled walking frames. For frames with a double bar across the front (see fig 1), simply push the caddy onto the bars until you hear a click. This indicates that the caddy is secured into position.

For frames with a single bar across the front (see fig 2), the additional clips are required to prevent the caddy rotating on the frame. Prior to attaching the caddy to the frame, the rear clips need to be secured to the caddy. To fit the clip, locate the circular pin into the circular hole at the rear of the caddy and slide along the channel. For standard frames, the clip should be angled outwards, for narrow frames, it is best if the clips are angled towards the centre of the caddy.

Now push fit the caddy onto the frame, as above, and slide the two clips along the channel so that they can be located onto the two front legs of the frame. Once the caddy is in place, it can be used with or without the tray or cup holder. To fit the tray, simply position on top of the caddy. There is no locking mechanism, it simply sits in position into the top of the caddy.

When the tray is in position, you may fit the cup holder, if required. Again, this simply sits inside the tray. The anti-spill mug cover can be used to prevent spillages. Note, the mug holder should always be used with the circular base positioned within the tray.

Restrictions for use

The Buckingham Walking Frame Caddy has been designed solely to carry light, everyday personal items and should not be used to carry pets, small children, fuel of any kind, large volumes of liquid, heavy tools, domestic appliances or equipment.

Do not carry items that overhang the tray or are larger than the pannier.

Do not carry hot drinks on the tray without the Buckingham Mug Holder.

Boiling drinks and teapots should not be carried using the caddy. Only for use on wheeled walking frames. Maximum carrying weight 2kg (4.4lb).

Cleaning

All parts of the caddy may be washed up to temperatures of 75°.

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