

# USER INSTRUCTIONS

## LINTON PLUS RAISED TOILET SEAT

M53163	2" (50mm) Non Retail Pack - No Lid	M53175	4" (100mm) Non Retail Pack - No Lid
M53564	2" (50mm) Retail - No Lid	P42469	4" (100mm) Retail - With Lid
P42458	2" (50mm) Retail - With Lid	P42446	6" (150mm) Retail - No Lid
M53576	4" (100mm) Retail - No Lid	P42471	6" (150mm) Retail - With Lid

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

### Maximum User Weight



## 1.0 INTENDED USE

The Linton Plus toilet seat's raised design makes sitting and standing from the toilet easier, with less strain on the user's joints. The cut-out seat design makes personal cleansing easier and distributes pressure more evenly away from the coccyx. The seat is adjustable and fixes securely to standard shaped toilets. Seats are available in 3 sizes and can be supplied with or without a lid.

## 2.0 SAFETY

The maximum user weight must not be greater than 30st (190kg).

Inspect the seat and fixings at regular intervals to make sure that it is safe and secure.

Do not use the seat unless you are sure it is safe to do so. If in doubt, contact your supplier for advice.

Inspect the seat and fixings at regular intervals for wear and damage.

If damage is found, stop using the seat and contact your supplier immediately for advice.



**Figure 1: Raised Toilet Seat Assembly**

## 3.0 UNPACKING

Refer to Figure 1. Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.


### Contents:

1. Linton Plus Raised Toilet Seat x 1
2. Support Bracket and Handwheel x 2

## 4.0 ASSEMBLY

### 4.1 Fitting the Support Bracket:

The support bracket and hand-wheel screw assembly must be fitted to the seat, before fitting to the toilet.

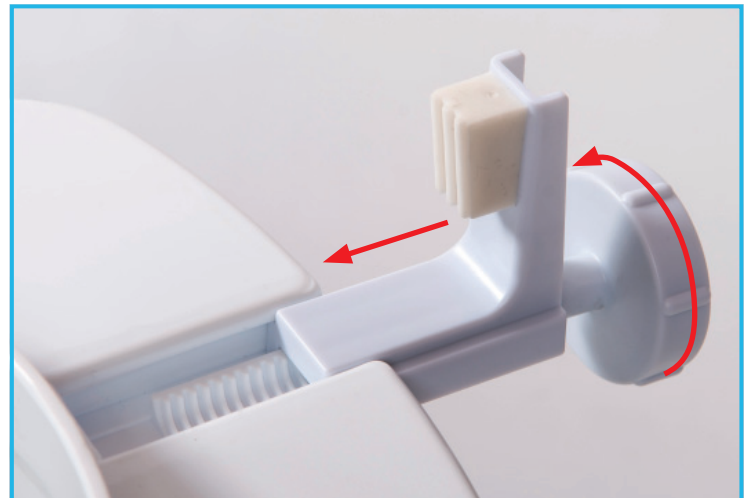

**Figure 2: Fitting the Support Bracket**

1. Refer to Figure 2. Slide the support bracket into the slot on the toilet seat.
2. Turn the hand-wheel clockwise until the threads engage.

**Note:** Do not tighten any further at this stage.

3. Repeat steps 1 and 2 for the opposite side.

### 4.2 Fitting to the Toilet:

1. Make sure that the existing toilet is safe and secure before you attempt to fit the product.
2. The existing toilet seat and lid must be lifted before the product can be fitted.
3. Check that the support brackets are securely in place before fitting.



**Figure 3: Seat Position**

4. Refer to Figure 3. Place the product onto the existing toilet, making sure that the front lip and support brackets are positioned outside of the bowl.
5. Check the seat is positioned evenly before securing to the bowl.
6. Secure the seat to the toilet bowl by turning the support bracket hand-wheels clockwise simultaneously, at equal amounts, on both sides.
7. You must check that the seat is fully secure before use. Apply reasonable force, front to back and side to side to check for movement.
8. If you find movement in the seat, re-tighten the support bracket hand-wheels until secure.

#### **4.3 Remove the Seat:**

1. To remove the seat, unscrew the support bracket hand-wheels in an anti-clockwise direction, at equal amounts, until the seat becomes free. Lift the seat off the toilet bowl.

#### **5.0 IN USE**

***WARNING: Do not stand on the seat under any circumstances, failure to comply can result in serious injury.***

1. We recommend that you check the seat at regular intervals. Make sure that the screws are tight and the seat remains evenly positioned on the toilet bowl.

2. If you think the seat has moved out of position, release the support bracket hand-wheels, by turning anti-clockwise, then reposition the seat on the bowl and tighten the support brackets, in accordance with Section 4.2.
3. If you remove the seat for any reason, it must be re-installed in accordance with Section 4.2. Failure to do so could compromise the safety of the user.

#### **6.0 CLEANING**

1. Clean regularly, using a general purpose, neutral detergent and warm water.
2. Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
3. Do not use any form of solvent as this will damage the surface and weaken the product and invalidate any warranty.
4. To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
5. These seats may be autoclaved on cleaning cycles up to 73°C.
6. Dry the seat thoroughly before use.

#### **7.0 PRODUCT DISPOSAL**

1. This product should be thoroughly cleaned prior to disposal.
2. This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

#### **8.0 PRODUCT GUARANTEE**

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel. Customer Services: 0345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)

M53163-UI-6