

# User Instructions

## PU Perching Stool Range

**L35555 - Adjustable Height Stool**

**L35567 - Adjustable Height Stool with Armrests**

**L35579 - Adjustable Height Stool with Armrests & Backrest**

**L35580 - Adjustable Height Stool with Armrests & Padded Backrest**

**L61049 - Extra Low Adjustable Height Stool with Armrests & Padded Backrest**

**L61050 - Extra Low Adjustable Height Stool with Padded Armrests & Padded Backrest**



### General Safety Warning

**Please ensure these instructions are fully read and understood before use. Retain these instructions in a safe place for future reference.**

#### 1.0 Intended Use

This perching stool is intended to be used to provide a stable, seated support, where a user is unable to stand for long periods, the seat slopes forward to allow the user to achieve a relaxed "perching" position when sitting and also to assist when rising.

#### 2.0 Safety Precautions

- Do not exceed the maximum user weight limit.
- Inspect product at regular intervals for signs of wear or damage
- Do not use if you see or suspect a fault.

#### 3.0 Unpacking

Carefully remove the Perching stool from its packaging and inspect for any signs of damage. If you suspect a fault, please contact your supplier.

#### 4.0 Height Adjustment

- To adjust the stool to the desired height
- Remove the plastic clips connecting the stools' legs and leg extensions.
- Slide the leg extensions to the required position, ensuring that all legs are equal in height.
- Replace the plastic clips ensuring that each clip is fully connected.

**Important:** Ensure the floor surface is level and even to avoid rocking.



#### 5.0 Product Care

##### 5.1 Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water
- Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water
- Dry thoroughly before use.

## Product Care - Continued

### 5.2 Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules
- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Check for general wear or damage.
- If you see or suspect a fault do not use – contact your dealer immediately

### 6.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**From the NRS Healthcare Range, you may also like:**

Nuvo Adjustable Bath Step  
Code No: M11077



Spa Curved Grab Rail  
M53527 355mm (14")  
M53539 480mm (19")  
M85553 620mm (24 1/2")



Combi Grabber  
N72358 660mm (26")  
N72369 813mm (32")



### NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business Park, Bardon Hill,  
Coalville, Leicestershire.  
LE67 1UB UK

Tel. Main Office: 0330 100 00 98

Tel. Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)



**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:  
**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**