

Product Technical Data Sheet 089

Atlas Advance Safety Belt

Product Range

P11449	Safety Belt — Small
P11450	Safety Belt — Medium
P11462	Safety Belt — Large

Product Description & Intended Purpose

The Atlas Advance Safety Belt is designed to provide support to the user of the Atlas Advance Transfer Aid over short distance transfers. It may also be used by two carers to assist the user rising from or lowering into a seated position, provided a suitable risk assessment is completed.

The user must be able to support their own weight and maintain balance for prolonged periods of time.

Product Features

- Wide Velcro fastening.
- Colour coded fixing points.

Technical Specification

P11449 : Hip Size 90cm - 105cm (35" - 41")

P11450 : Hip Size 100cm - 120cm (39" - 47")

P11462 : Hip Size 115cm - 140cm (45" - 55")

Maximum User Weight: 155 Kg (24 Stone)



Installation Information: *(Note: for detailed information refer to User Instructions)*

- Position belt around the waist of the user and fasten the Velcro.
- Attach the fixing straps to the buttons on the transfer aid handle.

Safety Precautions

- Do not exceed the maximum user weight of 155Kg (24st).
- Always perform a risk assessment prior to use.
- NRS recommend using the safety belt at all times for transfers over distances > 1—2 metres.
- Do not leave the user unattended when using the product.
- Do not leave in an area with unsupervised children present.
- Only for use with the Atlas Advance Transfer Aid.

PTDS089

Atlas Advance Safety Belt

Cleaning

- Hand wash at a low temperature using a mild detergent. Do not use bleach. Do not machine wash. Do not steam clean.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the belt and invalidate any warranty.
- Allow the product to thoroughly dry prior to use or storage.

Care & Maintenance

- Check the condition of the seams and stitching.
- Periodically inspect the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

Accessories / Related Products

N/A

FAQ's

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK