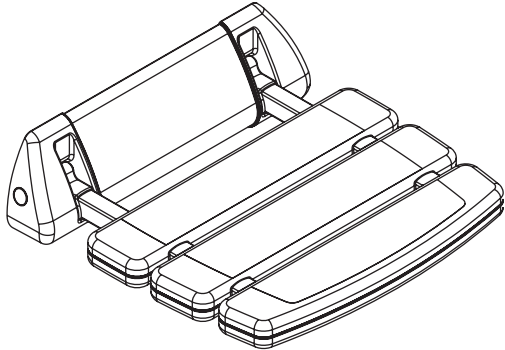
Serie 200

Safety
and comfort

PROVEX Industrie GmbH / S.r.l.
Industriezone Nord / 10 Zona Industriale Nord
I-39031 Bruneck / Brunico (BZ)
Fon: +39 0474 57 15 11
Fax: +39 0474 57 15 71
www.provex.eu
info@provex.eu

Provox - Garantiezusage

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Guarantee
The guarantee conditions for this product are as defined by our dealer in the country of sale. Please contact him if necessary. Guarantee and weight capacity only for standard products.

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GB

Cleaning and mainteance

- Do not use scouring or aggressive detergents, or petrol or thinner.
- Provox recommends a general household detergent or "Mister Proper". After cleaning rinse well with water.
- Provox recommends for lime deposit a vinegar essence (25%) or household vinegar.
- Do not use shower gels that are aggressive to polystyrol synthetic materials.

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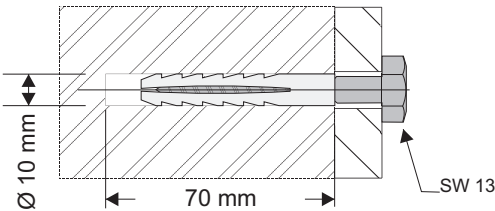
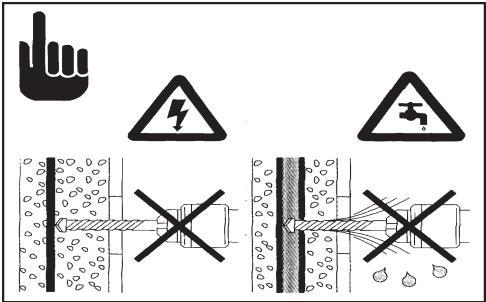
GB STANDART FIXING:
The following material is enclosed for standard fixing: Nylon Plug 10 mm Screw 8 x 60 mm
suitable for: Reinforced Concrete R250
For walls not listed above a Specialist in Fixing technology must be contacted.

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GB IMPORTANT POINTS:
The proMed shower seat has been designed to be fitted inside a shower cubicle or over a bath. IT MUST BE FITTED TO A SOLID WALL OR OTHER FIRM SURFACE. The wall face must be flat
The installer should ensure that the nature and construction of the wall will provide a firm fixing for all 4 screws supporting the shower seat.
FIXING HEIGHT:
The fixing height of the shower seat depends largely on the height of the user, it will usually be somewhere between 450 - 550 mm above the bottom of the shower tray or bath.
(A simple method of determining the shower seat height would be to measure the seat height of the users favourite / most comfortable chair, and fix the shower seat at a similar height.)

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