

## User Instructions

### Mobility Care Folding Ramp Range

**Maximum User Weight:**



#### General Safety Warning

**Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.**

#### Intended Use

This range of portable, aluminium folding ramps are designed to provide access for wheelchair users (and if appropriate, their carers') over thresholds, steps and curbs up to 6" high.

#### Instructions for Use

1. Your Mobility Care Folding Ramp arrives fully assembled and requires no additional assembly before use.
2. Check the ramp is free from damage before use.
3. With the ramp still folded place on the landing (higher) surface, ensuring that the top lip is resting securely on the landing surface and the lower edge of ramp is on the ground.
4. Unfold the ramp.
5. Ensure the ramp is stable and secure.

**L28228 - 915mm (3 Feet)**

**L28216 - 1220mm (4 Feet)**

**L28241 - 1520mm (5 Feet)**

**L28253 - 1830mm (6 Feet)**



6. If possible, fix the ramp to the threshold using fixing holes provided.
7. When in use try to keep the person/load in the centre of the ramp when pushing.
8. Do not exceed the maximum load weight of 300kg (47st).

#### Important Note:

To prevent the ramp from becoming dislodged during use, ensure the top lip overlaps the landing surface as far as possible. The surface holding the top lip needs to be substantial; make sure the surface and supporting structure is of sound construction.

Specification	L28228	L28216	L28241	L28253
Usable Length	915mm (36")	1220mm (48")	1525mm (60")	1829mm (72")
Overall Width	738mm (29")	738mm (29")	738mm (29")	738mm (29")
Usable Width	730mm (28¾")	730mm (28¾")	730mm (28¾")	730mm (28¾")
Folded Size	915 x 360 x 73mm 36" x 14" x 2¾"	1220 x 360 x 73mm 48" x 14" x 2¾"	1525 x 360 x 73mm 60" x 14" x 2¾"	1829 x 360 x 73mm 72" x 14" x 2¾"
Weight*	8.4kg	11.0kg	14.0kg	16.7kg

*\*Weights are approximate and may vary*

## Safety Precautions

When choosing a ramp, the length, and therefore the angle of the ramp depends upon:

- The height to be raised
- The type of wheelchair or scooter
- The weight of the wheelchair or scooter and the user
- The strength of the helper or user.

As a general guide, a raise of 1 in 12 is suggested when the user is seated on the wheelchair or scooter.

For loading an unmanned wheelchair or scooter into a vehicle, a higher gradient can be used, but this should not exceed 1 in 3. In addition to this, it is important that you refer to your vehicle's user guide for the proper degree of incline/decline and direction before using the ramp.

## Never exceed its recommendations.

- Only carry the ramp using the carry handle. Take care when lifting; follow standard manual handling procedures.
- Take particular care when using the ramp in wet/damp conditions.

## Care and Maintenance

Inspect the ramp regularly for signs of damage/ loose screws etc.

Do not use if the ramp is damaged.

Clean regularly, wipe over with a solution of mild detergent and water.

## Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

This product is from a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range visit:

[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk) or request a catalogue from our customer services department.

## NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business Park, Bardon Hill,  
Coalville, Leicestershire.  
LE67 1UB UK

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email:

[customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:  
**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**