

**N05370 - Corner Shower Stool**  
**N05382 - Corner Shower stool & Padded Seat**

**Maximum  
User  
Weight:**



**General Safety Warning**

Please ensure these instructions are fully read and followed. Failure to do so may result in injury to the user. Retain these instructions in a safe place for future reference.



**Intended Use**

The NUVO Shower stool is intended to be used in a shower to provide a stable, seated support, where a user is unable to stand for long periods (i.e. during showering).

**Unpacking**

The stool is supplied fully assembled. Remove from packaging and check for damage. If you see, or suspect a fault, do not use the product and contact your supplier.

**Safety Precautions**

- Ensure an even floor surface to avoid rocking.
- The three ferrules must be in contact with the floor at all times.
- Care should be taken when leaning forward
- Ensure the seat is fully located on the frame
- Do not exceed the maximum user weight limit
- Inspect at regular intervals for damage or excessive wear, if found or suspected, remove from use immediately and contact your supplier.

**Height Adjustment**

- The stool seat height is adjustable, to adjust to the desired height, firstly remove the plastic e-clips connecting the stools' legs and leg extensions.
- Slide the leg extensions to the required position ensuring that all legs are equal in height.
- Replace the plastic e-clips ensuring that each clip is fully fitted.

**Product Care: Cleaning**

- Clean regularly, use a general purpose, neutral detergent and warm water
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and again invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water
- This shower stool is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 73°C
- Dry the seat thoroughly before use.

**Product Care: Maintenance**

- Inspect the stool at regular intervals, ensure the product is stable and rigid.
- Check all fixings are fully tightened and in good condition.
- Check ferrules (feet) monthly for signs of wear.

**Product Guarantee**

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS Healthcare**

Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill,  
Coalville, Leicestershire. LE67 1UB UK

Tel. Main Office: 0330 100 00 98

Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

**NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:**  
**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**