

Other products in the NRS Healthcare range you may find useful when using this product:

L35324: Dycem Non-Slip Mat



The Adjustable Tilting Over Bed & Chair Table and Dycem Non-Slip Mats are from a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range please visit our website or request a catalogue.

NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill,
Coalville, Leicestershire,
LE67 1UB,
United Kingdom.

Tel. Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk

User Instructions

M66832 Adjustable Tilting Over Bed & Over Chair Table

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Intended Use

This stylish and versatile table can be used either over chairs or single beds. It has four lockable castors which makes manoeuvring and positioning the table very easy. It is height adjustable from 700 mm (27½") to 900 mm (35½") and is width adjustable from 900 mm (35½") to 1100 mm (43¼"). The table top can be tilted to any angle from flat to 55° by means of a hand operated locking gas strut. The top surface comprises two side tables which do not tilt, enabling the user to rest a drink while raising the centre surface. The side tables can be removed if not required. All surfaces can be easily wiped clean.



Unpacking

- This product is supplied flat packed and requires assembly before use.
- No tools are required.
- Remove the product from its packaging; Check for damage. If you see any, or suspect a fault, do not assemble the product; contact your supplier immediately.
- Check all parts are present - see contents list overleaf.

| Contents | |
|----------------------------|-----|
| Description | Qty |
| Table Top Assembly | 1 |
| Side Table | 2 |
| Adjustment Angle | 2 |
| Hand wheel (Locking Screw) | 4 |
| Leg Assembly | 2 |


Safety Precautions

- Do not overload the table top - Max 15 kg when table top is flat.
- Do not use the table to transport objects.
- It is advised to lock the castors before using the table.
- Take care to remove any unsuitable items before raising/lowering the table top.

Assembly Instructions


1 - Place the table top assembly on the floor, face down - see diagram 1

2 - Slide the shorter leg of the adjustment angle into the table top centre bar, to the required width, align the holes in the two parts and secure with hand wheel - see

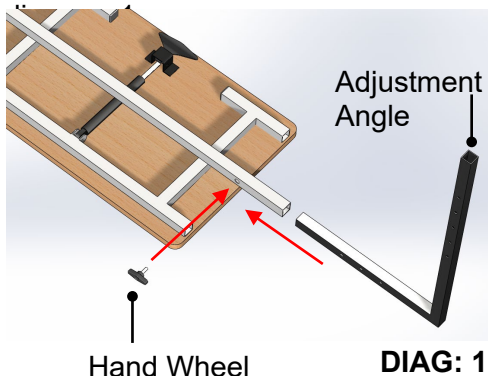

3 - Repeat step two for the other side.

4- Fit the leg assembly onto the Adjustment Angle, slide to the correct height. Align the holes and secure with hand wheel - see diagram 2.

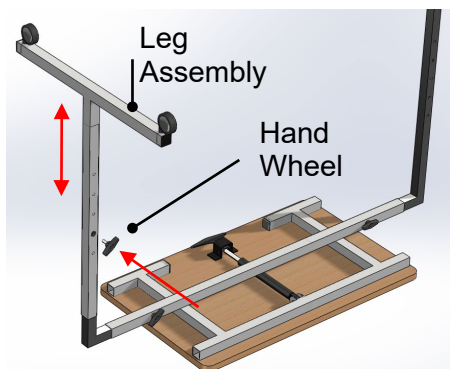


DIAG: 2


5 - Repeat step four, for the other leg.

6 - Turn the table over and lock the castors.

7 - Fit the side table by sliding the rectangular tubes into the corresponding tubes in the table top assembly, depress the spring snap clips, when sliding the tubes, push until the clips snap into position, see Diagrams 3 & 4.

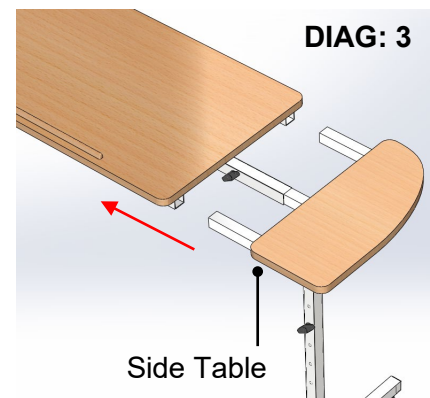


Side Table

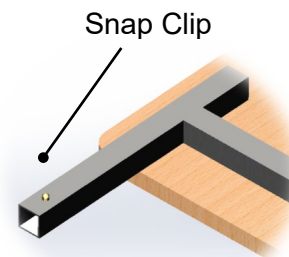

DIAG: 4

User Instructions

To Raise the Table top press the handle towards the underside of the table and raise/lower the top. Release the handle to lock in position See Diagram 5.

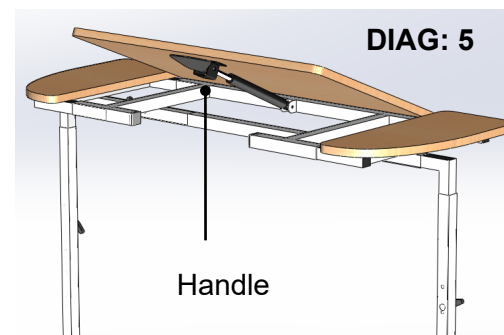

DIAG: 5

Cleaning

- Clean regularly, wiping down with a soft cloth and a mild detergent/disinfectant solution.
- Do not use abrasive cleaners.

Care & Maintenance

Inspect on a regular basis the stability and rigidity of the product, check all screws are tightened and in good condition.

Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered. This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only. Any guarantee issued by NRS Healthcare does not affect your statutory rights.