

Sensor Mat – Premium Class

Model No.: 1120G


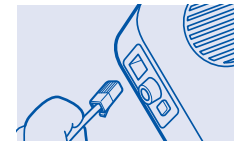
Operation Instruction:

1. Test the monitor device (i.e. Voice Alert Alarm) to ensure it is operating correctly.
Follow the monitor device instructions carefully, ensuring it is properly set up and has adequate battery power.

2. Connect the Sensor Mat to the monitor device and test the function and connection again.
Test by applying pressure on the Sensor Mat surface and then release.

3. Keep a record of the usage, by writing the patient’s name and the date you start and stop using this Sensor Mat in the area provided on the product. A suitable oil based permanent marking pen should be used.

4. Place the Sensor Mat on the bed, chair or wheelchair and position the Sensor Mat directly under the patient’s body weight.
This product can also be used as a Floor Sensor Mat (alarms when stepped-on).



Position under shoulder or buttocks or on wheelchair seat

Cautions and Warnings:

1. Read carefully and follow these instructions closely to minimise risk of injury or death to patients in your care.

2. This Sensor Mat is an aid only and is not intended to be a substitute for routine visual monitoring of the patient. Frequent routine inspections to ensure proper placement of the sensor mat, its connection and the monitor device (i.e. settings and battery condition) is essential. This Sensor Mat may not be suitable for high-risk patients.

3. This is a disposable item.
 - If the surface or cable becomes wet or damaged the mat should be replaced.
 - If after extended use any deterioration in performance of the mat is experienced (e.g. intermittent alarm) the mat should be replaced.

Care and Maintenance:

- Do not fold or bend the mat. Store flat.

• This mat may not function properly when used with a monitor device from another manufacturer. Use only a monitor device from the recommended manufacturer.

• Do not immerse the mat in any liquid/solution. Wipe clean only with a non-caustic sanitising solution.

• The mat should be cleaned before use, regularly during use and after use.

• To minimise the risk of infection single patient use only is recommended. Do not share the mat between patients.

• Periodically test the mat to ensure it is functioning properly. As with all products, wear and tear from excessive use will reduce performance.

• Do not attempt to repair or service the mat. If damaged it should be replaced.

Patient’s Name:

Made in China

Start Use Date	End Use Date

Also Available:

1120G

Sensor Mat Premium: 24 x 24 cm

1120C

Sensor Mat Premium: 37 x 24 cm

1120D

Sensor Mat Premium: 75 x 24 cm

332

Patient Alert Magnetic Alarm

345.3

Voice Alert Alarm

345W

Wireless Alarm Kit for Sensor Mats

1120

Alert Care Alarm with Remote Disarm