

User Instructions

Folding Comfort Footrest M12185

General Safety Warning

Please ensure these instructions are fully read and understood before installation/ use. Retain these instructions in a safe place for future reference.

Intended Use

The Folding Comfort Footrest is intended to provide a comfortable support for a user's legs whilst seated. The leg support pad is angle and height adjustable so that it may be positioned to meet the user's requirements and it may be folded away for storage.

Safety Precautions

Do not stand or sit on the Footrest, it is not intended to support full body weight.

Assembly

The product is supplied fully assembled. Simply open the base frame out until the strap is taught.

To adjust angle: Loosen hand wheel under cushion and tilt as required. Re-tighten hand wheel.

To adjust the height of the foot rest: remove the pin clips and slide out the tubes to the required height, before replacing the pin clips, ensuring they fit fully back onto the tube.

Cleaning Instructions

On the metal work: Remove any dirt or dust using a soft, damp cloth. Use mild detergent solution to remove stubborn stains. Dry with a soft cloth.



Manufactured in the UK for:

Nottingham Rehab Supplies

Clinitron House,
Excelsior Road,
Ashby Park
Ashby de la Zouch,
Leicestershire,
LE65 1NG
United Kingdom

Tel: 0845 120 4522

Int'l: +44 (0) 1530 418378

Fax: 0845 121 8112

Email: customerservice@nrs-uk.co.uk

This product is from a range of over 3000 daily living aids available from Nottingham Rehab Supplies. To view the full range visit www.nrs-uk.co.uk or request a catalogue from the above number.