

### Maximum User Weight



### General Safety Warning

Please ensure these instructions are fully read and understood before use. Retain these instructions in a safe place for future reference.

#### 1.0 Intended Use

This perching stool is intended to be used to provide a stable, seated support, where a user is unable to stand for long periods. The seat slopes forward to allow the user to achieve a relaxed "perching" position when sitting and also to assist when rising.

#### 2.0 Safety Precautions

- Do not exceed the maximum user weight limit.
- Inspect product at regular intervals for signs of wear or damage.
- Ensure the floor surface is level and even to avoid rocking.
- Do not use if you suspect a fault.

#### 3.0 Unpacking

Carefully remove the Perching stool from its packaging and inspect for any signs of damage. If you suspect a fault, please contact your supplier.

#### 4.0 Height Adjustment

- To adjust the stool to the desired height
- Remove the plastic clips connecting the stools' legs and leg extensions.
- Slide the leg extensions to the required position, ensuring that all legs are equal in height.
- Replace the plastic clips ensuring that each clip is fully connected.

#### NRS Healthcare

Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB UK  
Tel. Customer Services: 0345 121 81111 Email: customerservices@nrshealthcare.co.uk



#### 5.0 Product Care

##### 5.1 Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water
- Please check the suitability of the product before use. Do not use abrasive cleaners or scourers as this will permanently damage the surface and invalidate any warranty.
- Dry thoroughly before use.

##### 5.2 Maintenance

- Check that all fixing clips are inserted correctly.
- Check for general wear or damage.
- If you see or suspect a fault do not use – contact your dealer immediately

#### 6.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



**NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:**  
[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)