

Product Technical Data Sheet 022 M11077 Nuvo Adjustable Bath Step

Product Range

N/A	N/A
-----	-----

Product Description & Intended Purpose

The Nuvo Adjustable Bath Step is designed to assist users transferring into and out of the bath by reducing the step-over height. It can also provide assistance when using the toilet or sink in a bathroom environment.

Product Features

- Non slip surface provides grip, even when feet are wet.
- Stackable sections can be removed or added to achieve the desired height rise.
- Easy to clean plastic moulding.



Technical Specification

Dimensions:

- Length: 500mm (19 ½")
- Width: 395mm (15½")
- Heights: 55mm (2"), 77mm (3"), 100mm (4")
- Maximum User Weight: 317kg (50st)

Installation Information *(Note: for detailed information refer to User Instructions)*

- This product requires assembly before use.
- No specialist tools or skills are required to assemble this product.
- Assembly time is led than 5 minutes.
- For stacking instructions please see user instructions or visit

<http://www.nrs-uk.co.uk/nuvo-adjustable-bath-step.html>

Safety Precautions

- Do not exceed max. user weight of 317Kg (50st)
- The maximum step height is 175mm (7") using 6 step sections (2 packs). Do not stack more than 6 step sections together.
- Always ensure the bottom section of the step has 4 support pillars inserted in the base.

PTDS 022

M11077 Nuvo Adjustable Bath Step

Cleaning:

- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
- The product can be autoclaved on cycles up to 80°C.
- Allow the product to dry fully prior to use or storage.

Care & Maintenance

- Check that all support pillars are inserted correctly and any additional sections are securely fitted.
- Inspect the product regularly for signs of wear or damage. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

Accessories / Related Products

M24345: Support Pillars (Set of 4)

FAQ's

- What is the maximum height achievable with the step?
A single pack of steps will offer a rise of 2", 3" or 4". Two packs can be stacked together to achieve a maximum rise of 7". It is not recommended to stack the steps any higher than this.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK