
ANIMO

Safety and comfort




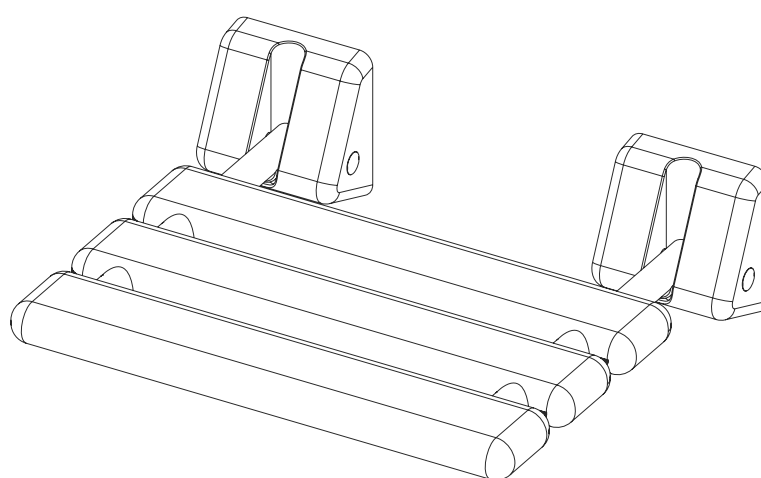
Provex®

comfort in the bathroom

**max
120 kg**

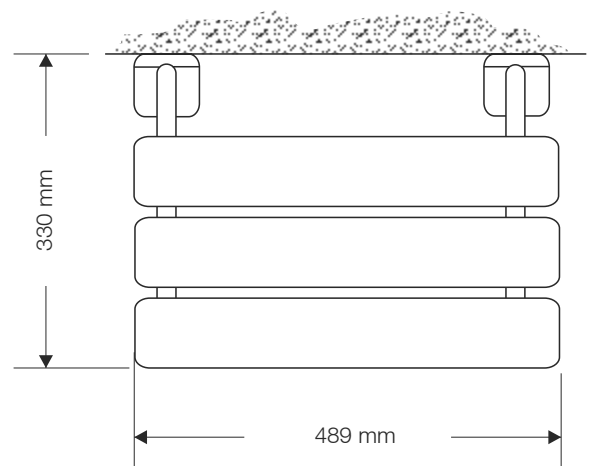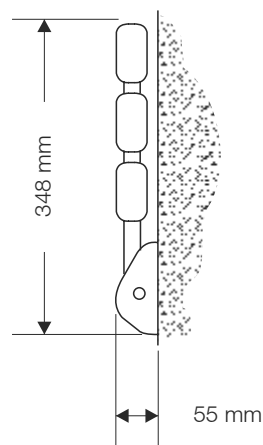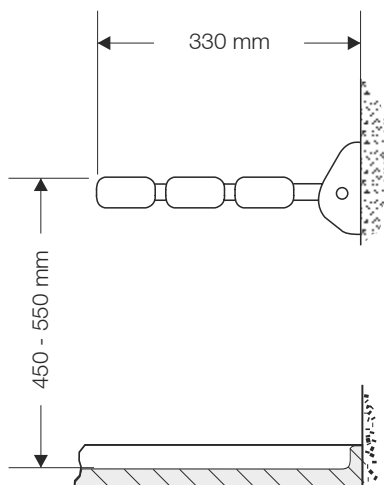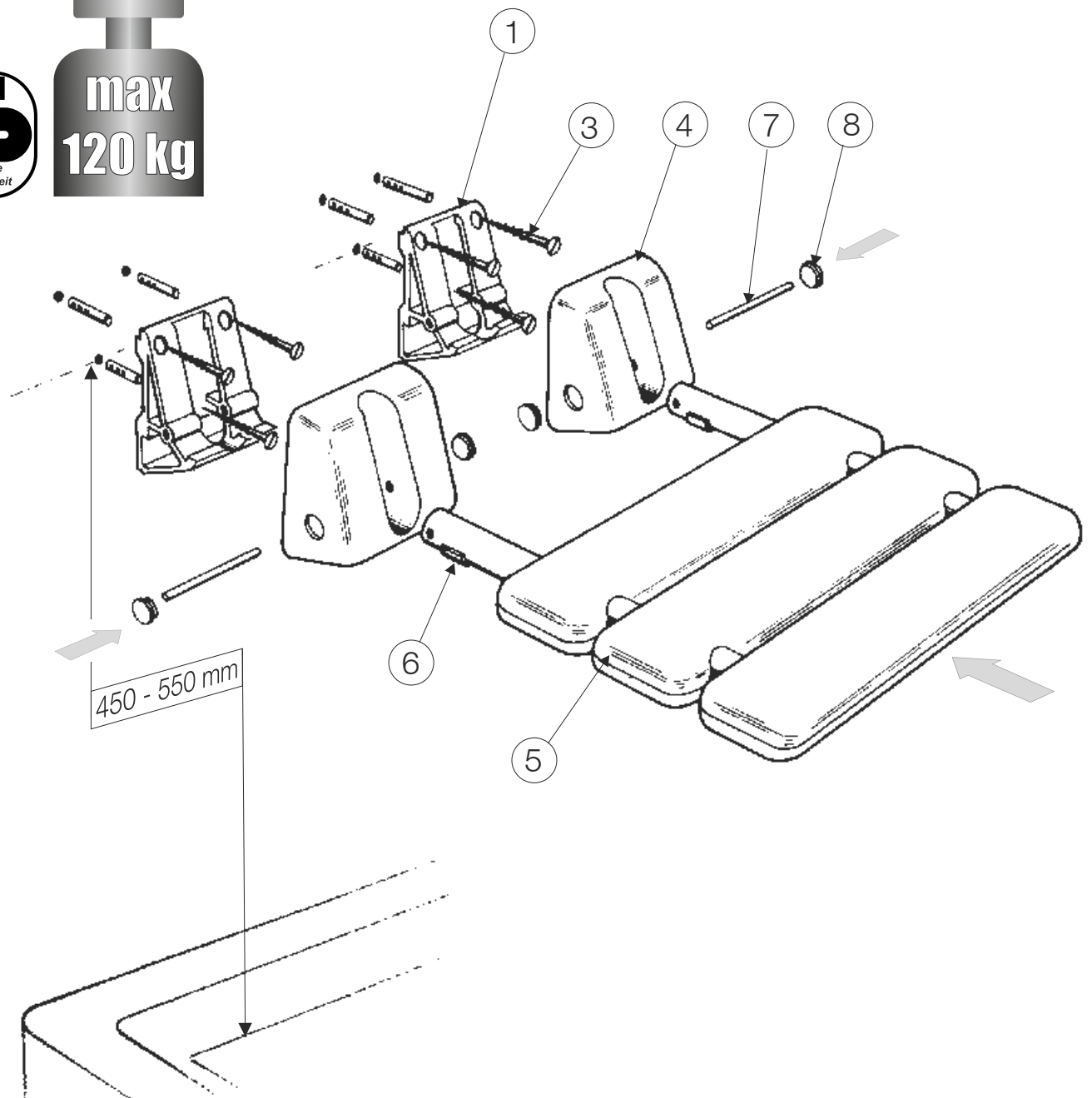D MONTAGE:

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GB INSTALLATION:

Before fitting ensure that the position of the shower seat not interfer with the plumbing or electrical fitting.

- a) Affix the drill template to the wall horizontally, as per diagram. As marked on the template, drill holes using an 8 mm drill, then insert bolts (2). Remove templates.
- b) Fix the wall socket (1) to the wall, using the 3 screws (3) provided per wall socket.
- c) Ensure that the fixing screws are fastened, then place the two sockets (4) to the wall.
- d) Push the two arms holding the seat into the sockets and insert one of the hinge pins (7) in the socket and through the arm of the shower seat. Repeat this with the other hinge pin.
- e) Cover the end of hinge pins with the caps.

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GB IMPORTANT POINTS:

The shower seat has been designed to be fitted inside a shower cubicle or over a bath. IT MUST BE FITTED TO A SOLID WALL OR OTHER FIRM SURFACE. The wall face must be flat.

The installer should ensure that the nature and construction of the wall will provide a firm fixing for all screws supporting the shower seat.

Max. user weight: 120 kg

FIXING HEIGHT:

The fixing height of the shower seat depends largely on the height of the user, it will usually be somewhere between 450 - 550 mm above the bottom of the shower tray or bath.

A simple method of determining the shower seat height would be to measure the seat height of the users favourite / most comfortable chair, and fix the shower seat at a similar height.

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PROVEX Industrie GmbH / S.r.l.