

User Instruction for the Stackable Commode – Fixed Height & Arms

T60 (2013 version with enhanced features) Brown Frame


T304 (2013) Blue Frame & Upholstery

T317 (2013) Reduced Height with Brown Frame


1. Introduction

The Stackable Commode is designed to enhance user comfort with heat sealed padded upholstered back and an oval seat. The commode is adjustable height (see product specification's) below at 25mm intervals. The feet are none-marking, none-slip for extra user safety. The potty has a capacity of 5 litres

All structural joints are fully welded and the toilet seat clips on and off. These features are designed to aid product hygiene by reducing the possibility of bacterial entrapment. The steel framework is also coated in an anti-bacterial powder coated finish. The clip-on design toilet seat aids maintenance as no tools are required to change a seat.

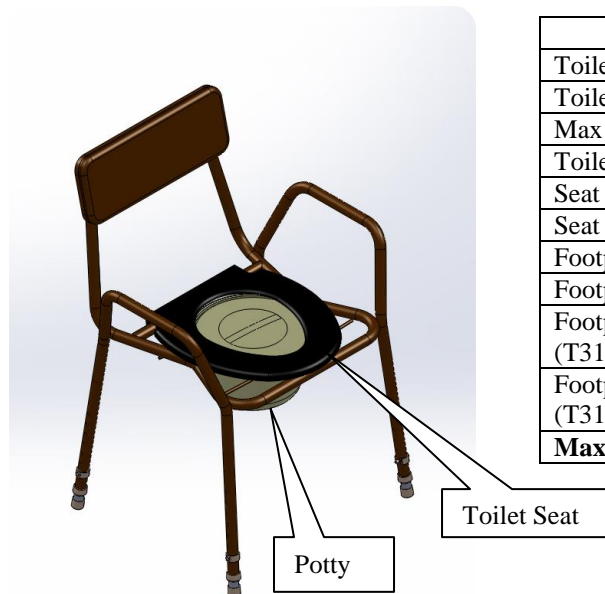

Product specification	
Toilet seat height (T60 & T304)	450mm – 600mm
Toilet seat height for the (T317)	405mm – 550mm
Max width between arms	460mm
Toilet seat depth	355mm
Seat to top of arms	180mm
Seat to top of back	330mm
Footprint min @ 450mm Ht	475mm x 435mm
Footprint max @ 600mm Ht	500mm x 465mm
Footprint for the min @ 405mm Ht (T317 only)	470mm x 425mm
Footprint for the max @ 550mm Ht (T317 only)	490mm x 455mm
Maximum user weight	159 KG (25 Stone)

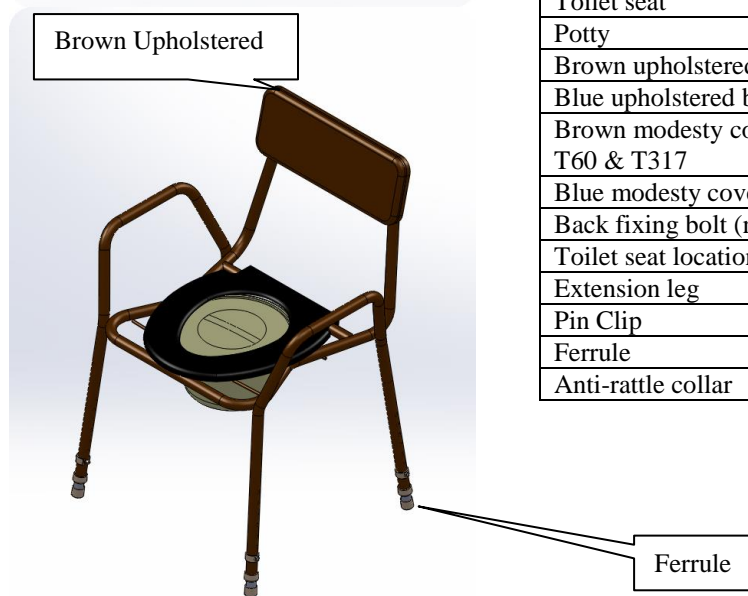

Parts list	
Description	Part No
Toilet seat	71SBS
Potty	71019A
Brown upholstered back - T60 & T317	67019B/2
Blue upholstered back - T304	67019BB
Brown modesty cover (not shown) - T60 & T317	67019S
Blue modesty cover - T304	67019BS
Back fixing bolt (not shown)	68M6/30
Toilet seat location clips (not shown)	69SF
Extension leg	56C319P
Pin Clip	63PC
Ferrule	69FR7
Anti-rattle collar	74GAR/2

User Instruction for the Stackable Commode – Fixed Height & Arms

T60 (2013 version with enhanced features) Brown Frame

T304 (2013) Blue Frame & Upholstery

T317 (2013) Reduced Height with Brown Frame

2. Contents

- Stackable Commode
- User Instructions (this leaflet)
- Batch label – This will be found on the product and will include the product code as well as a number. In the event of needing to contact the manufacturer about this product always quote the Batch Number.

3. Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete.
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

4. Correct Use

- Remove the vinyl modesty cover to use the commode.
- Replace the vinyl modesty cover **note: - this is not designed to be sat on as it is not fixed to the frame it is for cosmetic purposes only.**
- Please empty the potty at regular intervals and be aware of its maximum capacity.
- **Do not attempt to move the commode whilst occupied.**


Locating the Potty:

- To locate the potty in the commode, remove the loose modesty cover and raise the toilet seat to a vertical position resting it against the backrest.
- By holding the potty handle in the vertical position, place the potty, lid uppermost on the support brackets ensuring an accurate location of the wide brim on the brackets.
- Remove lid, fold the potty handle down and lower the toilet seat into the horizontal position.

Removing the Potty:

Once the commode has been used, repeat the process in the reverse order ensuring that the lid is correctly placed to avoid possible spillage.

Height Adjustment:

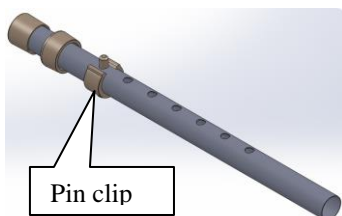

- Remove the pin clip from each of the legs
- Select the required height of the frame by gently sliding each extension leg in or out.
- When desired height is found refit the pin clip into the appropriate hole in each extension leg
- Ensuring it goes fully through. Ensure that all legs are set to the same height.

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- The toilet seat and retaining clips can be removed to aid cleaning. **This feature is an added benefit over traditionally fixed toilet seats as it ensures all surfaces are fully accessible for cleaning.**



User Instruction for the Stackable Commode – Fixed Height & Arms

T60 (2013 version with enhanced features) Brown Frame

T304 (2013) Blue Frame & Upholstery

T317 (2013) Reduced Height with Brown Frame

- Do not use an abrasive cleaner as this may damage the frame and nullify your warranty.
- It is recommended that the potty is thoroughly cleaned after every use, using an appropriate cleaning agent.
- Remove legs for cleaning.

Use of power/jet washing & industrial washing machines:

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependant on the manufacturer's instructions.

6. Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for users of a maximum of 159kg/25stone in weight.
- We recommend a full risk assessment should be carried out before use.
- Under no circumstance should the commode be moved whilst occupied.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage **however we can advise that this product has been designed, manufactured and independently tested in the UK (including cyclic testing to represent real life usage).**

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)





**User Instruction for the Stackable Commode – Fixed
Height & Arms**
T60 (2013 version with enhanced features) Brown Frame
T304 (2013) Blue Frame & Upholstery
T317 (2013) Reduced Height with Brown Frame

E-mail: cefndy.sales@denbighshire.gov.uk
