

Maximum User Weight



General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

1.0 Intended Use

The adjustable back support is intended to support a single user at a chosen inclined position whilst seated in bed. It is fitted on top of the mattress and underneath any supporting pillows.

2.0 Safety Precautions

Ensure the supporting arm is correctly fitted into the location points before leaning back onto back support.

Ensure bed is either fitted with a fixed head board or that the bed is positioned against a wall, so that the back support cannot slip backwards during use.


Periodically check the stability and rigidity of the back support and, where applicable, that screws are tightened and in good condition.

3.0 Cleaning Instructions

Remove any dirt or dust using a soft, damp cloth. Use mild detergent solution to remove stubborn stains. Dry with a soft cloth. For disinfection in hospital environments, please follow hygiene specialist's recommendations.

NRS Healthcare

Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK
Tel: 0845 120 4522 Email: customerservice@nrs-uk.co.uk

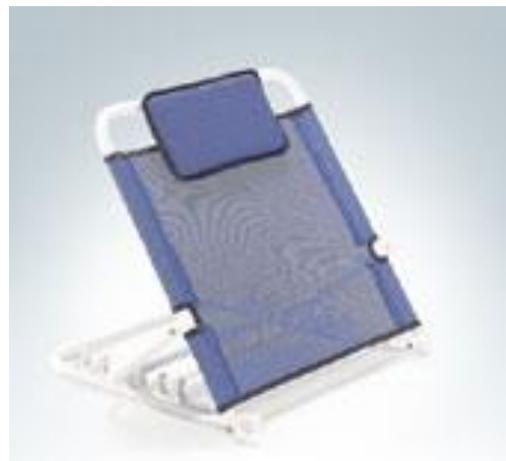

4.0 Fitting Instructions

To view a video showing how to install and use this product go to : <http://www.nrs-uk.co.uk/adjustable-back-rest.html>

1. Place on top of mattress (and any relevant bedding), with rear edge of base against the headboard/wall.
2. Adjust angle of back support to desired position.
3. Place pillows against back support as required.
4. Test to ensure back support is firm and cannot move backwards.

The back support is then ready for use.

5.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

www.nrshealthcare.co.uk