

**Maximum  
User Weight**



### General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

### 2.0 Intended Use

The Garden kneeler provides both a comfortable kneeling and sitting position when carrying out tasks at, or around, floor level. The sturdy arms are designed to assist getting up from a kneeling and sitting position. The product is lightweight and folds away for easy storage.

### 3.0 Safety Precautions

- Always ensure both handles of the kneeler are locked in position before use
- Do not exceed the maximum user weight
- Always place the kneeler on level, solid ground
- Always raise using the centre part of the handle

### 1.0 Unpacking

Remove packaging and check product carefully before use. Do not use if you see or suspect a fault.

Product is delivered folded, Instructions for assembly are shown overleaf.



Diagram:1 Product Folded



Diagram: 2  
Kneeler Option

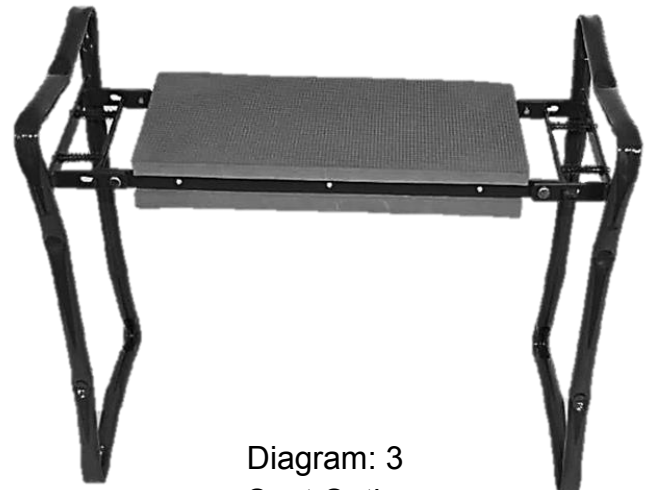


Diagram: 3  
Seat Option

### NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK

Tel: Head Office: 0330 100 0098

Tel: Customer Services: 0345 121 8111

Email:

customerservices@nrshealthcare.co.uk

**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)

#### 4.0 Assembly

- To unfold the kneeler, raise each leg in turn, outwards, as shown in diagram:4
- When in the fully upright position the legs will “click” into place, as shown in diagram:5
- Check the legs are locked and do not move from side to side before use



- To fold, place product in the kneeler position
- Grasp the handle side bar and sprung loaded locking rod and squeeze until the rod becomes dis-engaged from the locking slot. See Diag:6. The handle is now free to move
- Fold the handle inwards until it is resting on the seat pad, see Diag:7
- Repeat for the other handle

Diagram: 6



Sprung  
Locking  
Rod

Handle Side  
Bar

Diagram: 7



#### 5.0 Cleaning

Clean regularly, wiping down with a mild detergent /disinfectant solution.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage surfaces and invalidate any warranty.

#### 6.0 Product Disposal

Product should be cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

#### 7.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.