

USER INSTRUCTIONS

MOWBRAY LITE PLUS

N53909

Mowbray Lite Plus

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

To assist those who have difficulty in lowering onto or rising from a standard toilet. The comfortable plastic moulded seat offers support in a natural position. The seat lid adds discreet style and helps to improve cleanliness and hygiene. An optional floor fixing kit is also available (F11572).

2. SAFETY

WARNING: The maximum user weight must not be more than 190 kg (30 st).


DO NOT: Attempt to stand or climb on the product.

MAKE SURE: All legs are set at equal height to avoid rocking.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3.1 Contents:

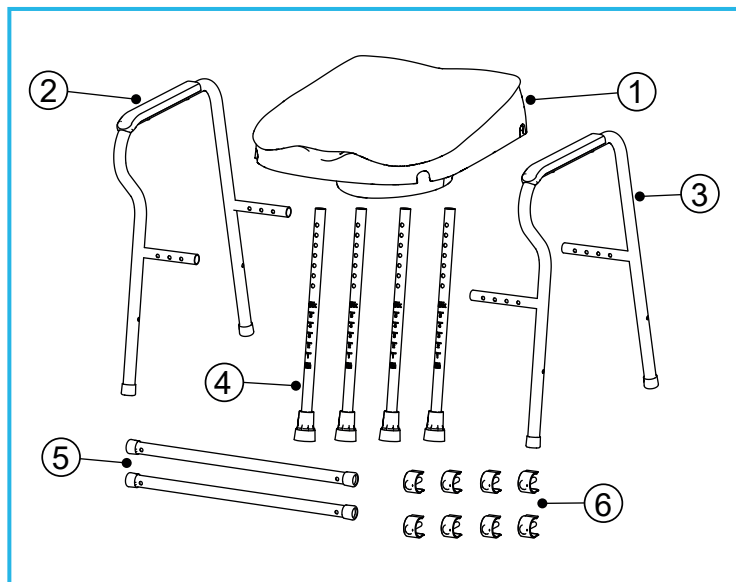


1. Toilet Seat & Lid x 1
2. RH Frame Assembly x 1
3. LH Frame Assembly x 1
4. Leg Extensions x 4
5. Crossbar x 2
6. E-clip x 8

4. ASSEMBLY/INSTALLATION

4.1 Fitting the Crossbars:


1. Slot the 2 x crossbars onto the side-frame assemblies and adjust to the correct width for the toilet and the user.

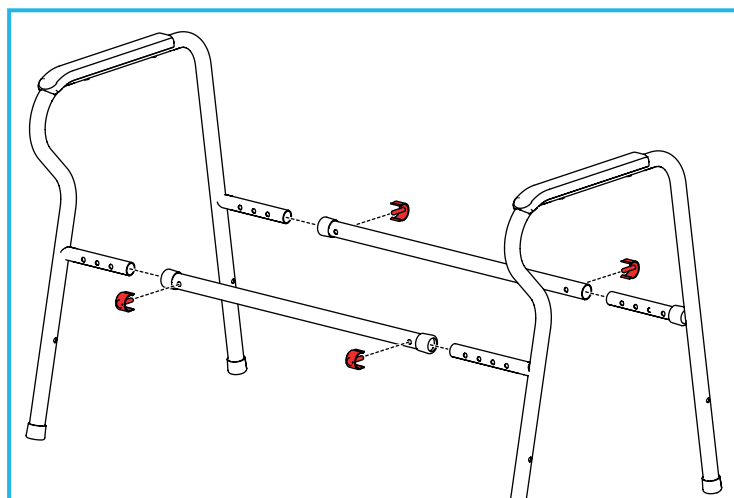

2. Align the holes and secure with the 4 x E-clips.

Note: Make sure that the side-frames and crossbars have been set to the same width/adjustment holes.


4.2 Fitting the Leg Extensions:

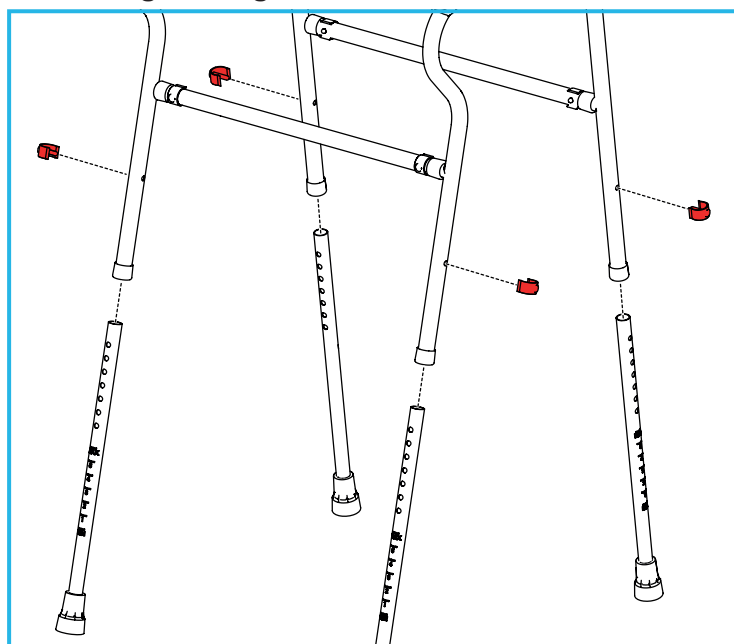

3. Slot the 4 x leg extensions into the side-frame assemblies and adjust to the correct height for the toilet and the user.
4. Align the holes and secure with the 4 x E-clips.

4.3 Fitting the Seat

Fit the seat to the frame in the correct position, with the front of the seat and curve of the armrests facing forwards.

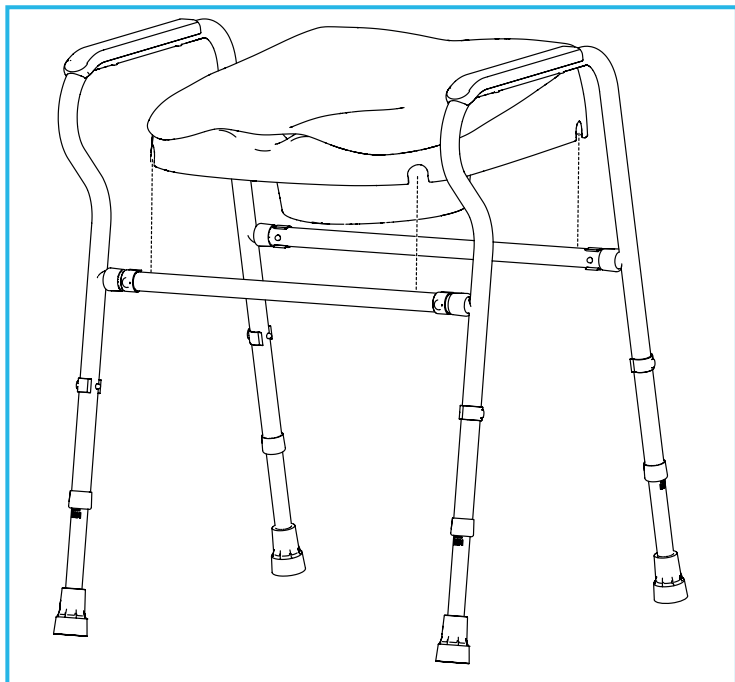

Push down firmly on the seat until it snaps into position on both the front and rear cross bars.

5. INSTALLATION

5.1 To install the Mowbray Lite Plus:

Place the frame over the toilet ensuring that the internal splash guard is central over the toilet bowl.

6. IN USE

Adjust the height and width for the toilet and user before installation.

Note: Set the side frame assemblies to the same width.

Note: Make sure the seat and toilet frame are fully secure before use.

6.1 Adjust the Height

1. Remove the 4 x leg extension E-clips.
2. Slide the leg extensions to the required height, align the holes and replace the E-clips.

6.2 Adjust the Width

1. Remove the 4 x crossbar E-clips.
2. Slide the side frame assemblies to the required width, align the holes and replace the E-clips.

6.3 Fitting the Optional Floor Fixing Kit

Please refer to the user instructions provided with the optional floor fixing kit (F11572).

7. MAINTENANCE

Check that all fixing clips are inserted correctly and that there is no excessive movement.

If a floor fixing kit has been fitted, check the fixing screws are secure.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

8. CLEANING INSTRUCTIONS

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use/storage.

9. ACCESSORIES & SPARE PARTS

F11572 - Floor Fixing Kit

N43218 - E-clips x 4

For further information please visit our website or call Customer Services on the details below.

10. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

11. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk