

# USER INSTRUCTIONS

## SWEDISH BATH RAIL

P23499 Bath Rail (White)

P42860 Bath Rail (Chrome)

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

## 1.0 INTENDED USE

The Swedish Bath Rail is designed to help steady the user while transferring into or out of the bath. It can be fitted to most standard baths.

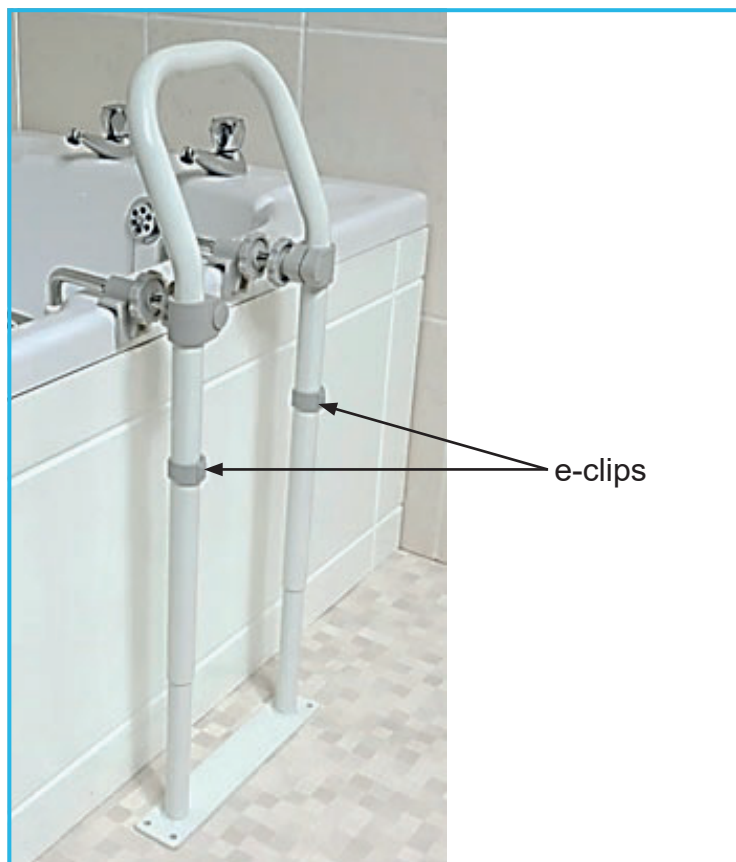


Figure 1: Swedish Bath Rail in Use

## 2.0 SAFETY

1. The maximum user weight must not be greater than 23st / 150kg.
2. Always check clamps and hand wheels are tight and that floor fixings are secure prior to use.
3. Not suitable for use by children.

## 3.0 UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### Maximum User Weight



## 4.0 INSTALLATION

**Warning:** This product may not fit securely to certain types of unusually shaped baths, including roll tops. Please check the suitability before installation.

**Warning:** Take care when installing onto fibreglass or plastic baths. DO NOT over-tighten, as the clamps may damage the bath.

Clean the bath wall thoroughly in the area where the rail is to be fitted.

### 4.1 Adjust the height of the rail:

1. Refer to Figure 1. Remove 'e-clips' and slide the rail to a height which is comfortable for the user when getting into and out of the bath.
2. Align the holes and re-insert the 'e-clips'.
3. Choose the desired position for the bath rail at the side of the bath.

**Note:** The handle should angle over the bath, see Figure 1.

### 4.2 Fit the Clamps to the Bath:

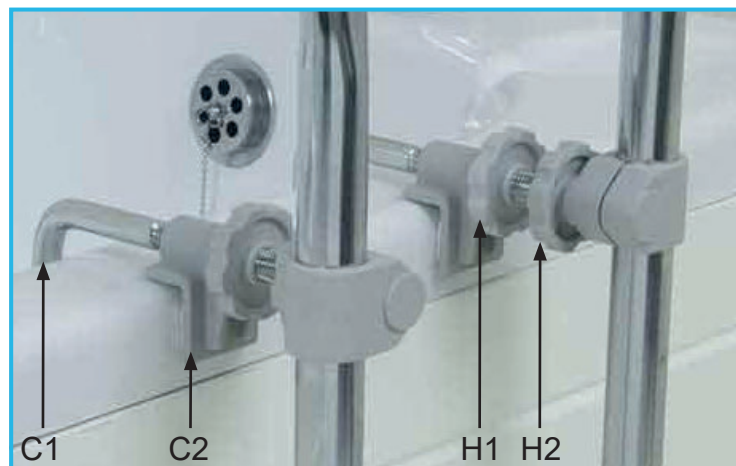


Figure 2: Clamps and Handwheels

1. Refer to Figure 2. There are two clamps. For each clamp, loosen handwheel H1 by turning anti-clockwise to open the clamp jaws C1 & C2.
2. Loosen handwheel H2 by turning anti-clockwise to allow the clamp mechanism to slide up and down the rail tube.
3. Slide the clamps into position such that C1 is on the inside of the bath edge and C2 is on the outside.
4. Tighten handwheel H2 by turning clockwise to secure the clamp mechanism at the correct height.
5. Tighten handwheel H1 by turning clockwise to close the clamp around the edge of the bath.

**Warning:** Do not over-tighten as the clamps may crack the bath.

6. Ensure the base plate is sitting flat on the floor and mark the position of the holes.
7. Remove the bath rail, by slackening the handwheel H1, on both clamps, then lift clear of the bath.
8. Drill and plug the holes in the floor.

**Note:** Fixing screws are not included. Choose a fixing method suitable for the construction of the floor, seek professional advice if you are unsure. A total of 4 screws must be used.

9. Replace the bath rail, and re-fit as previously described.
10. Fix to the floor.
11. Before use, check that the bath rail is secure by pulling backwards, forwards and side to side. There should be no excessive movement.

## 5.0 CLEANING

1. Clean the frame regularly, using a general purpose, neutral detergent and warm water.
2. Please check the suitability of the cleaning products before use.

3. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
4. To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
5. Always allow the product to dry thoroughly prior to use or storage.

## 6.0 MAINTENANCE

1. Check the clamps are correctly fitted and all hand wheels are tight.
2. Check the floor fixing is secure and that the bath rail has no excessive movement.
3. Periodically check the bath rail for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

## 7.0 PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 8.0 PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



**NRS HEALTHCARE** Sherwood House,  
Cartwright Way, Forest Business Park, Bardon  
Hill, Coalville, Leicestershire, LE67 1UB, UK.



**NRS HEALTHCARE** 4C Altona Road,  
Blaris Industrial Estate, Lisburn, Co  
Antrim, Northern Ireland BT27 5QB

Tel: Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)