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**P23488**  
One size fits all.

### Pack contains:

- 2 x leg sleeves (green).
- 1 x arm sleeves (blue).
- 1 x flat sheet in a Handy Storage Bag (yellow).

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### Benefits and Features

- Excellent sliding capability.
- Extremely versatile and flexible.
- Reduces/removes the effects of friction on the skin.

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### Uses

- Help carers put trousers/pyjamas on a bed bound person/child or someone who requires a lot of assistance.

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### Used by:

- Occupational therapists
- Mental Health teams
- District Nurses
- Community Nurses
- Carers at Home
- Children's carers

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### How To Use:

 Unravel the yellow slide sheet under the person you are caring for, then slide the two "leg" sleeves over the person's legs. (The slide sheet enables these leg sleeves to slide on easily). Then you can easily slide the person's trousers/pyjama trousers on and remove the leg sleeves by pulling gently from the foot end. Use the arm sleeves in the same way to slide on shirts, blouses, jumpers etc.

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### Washing/Cleaning Instruction:

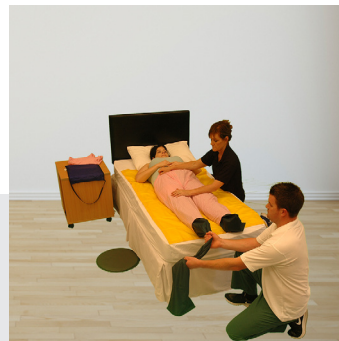
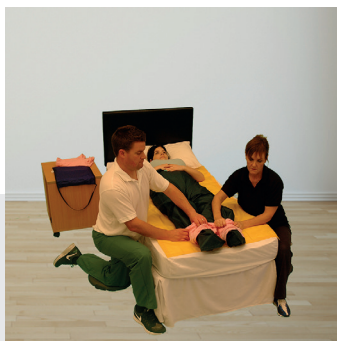
Washable models can be washed at 74°C. - Drip dry or tumble dry on a low/medium heat. - Can be autoclaved. - Do not iron. - Do not use in chlorine bleach.

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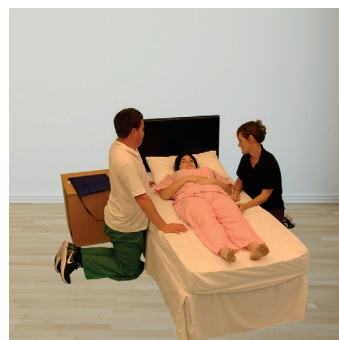
1. Firstly, insert the flat yellow slide sheet under person to be dressed.
2. Using the Green leg sleeves, gently pull these over the patients legs, the yellow slide sheet underneath will facilitate this.
3. Gently slide pyjamas, trousers or jeans over the green slippy leg sleeves.
4. From the foot end, gently pull and remove; first one slippy leg sleeve, and then the other.

1. Slide the arm sleeve gently over the patient's arm and up towards the shoulder, sliding the extra fabric under the patients back area. Use a flat upwards palm and the yellow slide sheet underneath to assist you.
2. Slide the pyjama/blouse or jumper sleeve over the blue slippy arm sleeve.
3. Once you have the garment in the required position, remove the blue arm slippy sleeve, carefully.
4. Repeat with the second arm and then remove the blue slippy sleeve.
5. Finally, remove the yellow slide sheet and, together with the leg and arm sleeves, put everything back in the pack.