

Product Technical Data Sheet 028

NRS Leg Lifter Range

Product Range

F19571	Single Handle Leg Lifter
L33194	Dual Handle Leg Lifter

Product Description & Intended Purpose

These leg lifters offer greater mobility and independence to the user providing support for lifting stiff or immobile legs. Particularly useful when getting in and out of bed, a wheelchair or car, also for repositioning legs when sitting.

Product Features

- Stiff polyester webbing for durability.
- Reinforced foot loop.
- Single or double handle for added control and comfort.



Technical Specification

Dimensions

- **Length** —F19571: 865mm (34")
L33194: 865mm (34")

Product Weight: approx. 90g (approx. 3oz)

User Instructions: *(Note: for detailed information refer to User Instructions)*

- Ensure the foot loop is open wide, mould it to roughly the correct shape and size for your foot.
- Hold the handle (F19571) or handles (L33194) and use them to help you hook the foot loop over your foot.
- Once your foot is comfortably inside the loop, use the handles to lift your leg up and guide it to the new position.
- Once your leg is in the desired position remove your foot from the loop and repeat the process for your other leg.

PTDS028

NRS Leg Lifter Range

Safety Precautions

- Do not leave product on the floor, it could become a trip hazard.
- Ensure foot is sufficiently inside the loop before attempting to raise leg.
- Take care when putting strap around wrists not to get it tangled.
- Not suitable for use by children.

Cleaning

- Clean regularly, use a general purpose, neutral detergent and warm water.
- Hand wash at low temperatures using a mild detergent. Do not machine wash.
- Allow product to fully dry prior to use/storage. Do not tumble dry.

Care & Maintenance

- Periodically inspect the product for signs of wear or damage, pay particular attention to the seams and stitching.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of action.

Accessories / Related Products

None

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville, Leicestershire. LE67 1UB UK