Using an ultra slippery material, these washable slide sheets allow health care professionals and carers to position and transfer patients with ease. Patient and user friendly, they are lightweight and durable, easy to insert and remove and can be used for repositioning up and down in bed, turning, rotating and transferring (dependant on size). Choose from one of seven sizes, including a small version for placing under heels and ankles or pillows. Machine washable up to 74 degrees, low temperature tumble dry.

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Benefits and Features

- Excellent sliding capability.
- Extremely versatile and flexible.
- Reduces/removes the effects of friction on the skin.

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Uses

- Repositioning up and down beds.
- Repositioning up and down on trolleys.
- Turning patients from one surface to another.
- Sitting up.

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Used by:

All areas of healthcare involved in repositioning people including:

- The patient/client unassisted
- The patient/client assisted
- Paramedic/Emergency Services
- Community-based staff
- General Ward staff
- Nursing and Residential Home staff
- Theatres and Radiology
- Disabled Units

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Washing/Cleaning Instruction:

Washable models can be washed at 74°C. - Drip dry or tumble dry on a low/medium heat. - Can be autoclaved. - Do not iron. - Do not use in bleach.

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Good Practice Points:

- Make sure that the brakes are on the bed/trolley before commencing a movement.
- Make sure that the sheet is inserted correctly.
- Insert the sheets by either:
 - a.) 'log-rolling' the patient/client to one side (or top to bottom) b.) unrolling directly beneath the patient.
- Wherever possible, ask/help the patient/client to help themselves.
- Removal is best achieved by turning the outermost corners in on themselves and peeling the sheet out using a wriggling, alternate-arm action. This effectively 'peels' the sheet out without creating any shearing effects

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