

## Product Technical Data Sheet 091 Mobility Care Outdoor Half Step

### Product Range

P22964 Outdoor Half Step - Black

### Product Description & Intended Purpose

The Outdoor Half Step provides a stable platform to help users with reduced mobility negotiate doorways and thresholds. It reduces the height of steps enabling greater access into and around the home.

### Product Features

- Moulded, durable plastic step.
- Four height adjustable feet.
- Rubber mat and textured surface for added grip.
- Lightweight for carrying and positioning.



### Technical Specification

#### Dimensions

- Length: 730mm (29 $\frac{3}{4}$ "
- Width: 425mm (16 $\frac{3}{4}$ "
- Depth: 110mm (4")
- **Maximum User Weight** : 190kg (30st)
- **Product Weight**: approx. 5.5kg (12lb)

### Installation Information: *(Note: for detailed information refer to User Instructions)*

- The step is delivered without the feet attached. These must be assembled prior to use.
- A spanner is supplied to assist with assembly.
- Assembly time is approximately 5 minutes.

### Safety Precautions

- Do not exceed maximum user weight 190Kg, 30st.
- Check that all four feet are flat and level on the ground prior to use.
- Always stand centrally on the step.

# PTDS091

## Mobility Care Outdoor Half Step

### Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- Allow the product to dry fully prior to use or storage.

### Care & Maintenance

- Periodically check the feet are securely fitted and the step is level.
- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the feet.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

### Accessories / Related Products

#### Spare Part

- P22976 Spare Feet - Set of 4

#### Accessories

- M20259 - Prima Outdoor Grab Rail

- Flat End Steel Grab Rail - various colours and sizes available.
- Round Flanged Steel Grab Rail - various sizes available.

### FAQ's

None

### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK