

USER INSTRUCTIONS

ECONOMY TOILET FRAME RANGE

M00869	Economy Toilet Frame 17" (430mm)	M87203	Economy Toilet Frame 19" (480mm)
M00870	Economy Toilet Frame Width Adjustable		

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

The Economy Toilet Frame is designed to assist users when lowering onto and rising from the toilet. It provides comfortable and stable support when getting on and off the toilet and is suitable for use in conjunction with a raised toilet seat.



M00869



M00870

2. SAFETY


Warning: The maximum user weight must not be more than 190Kg (30st).

Do not attempt to stand or climb on the product. Ensure all legs are set at an equal height to avoid rocking.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. HEIGHT ADJUSTMENT

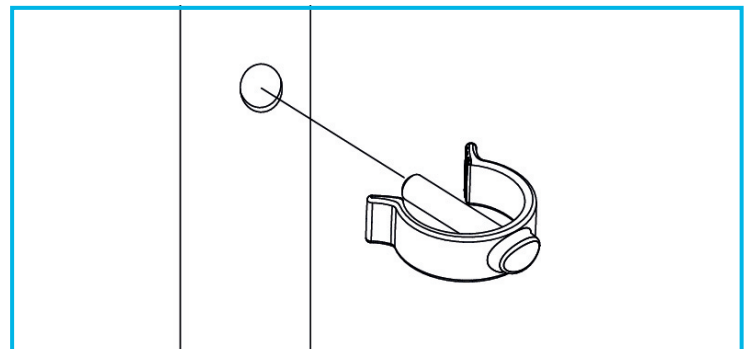

Remove the 'e-clips' and slide the legs to the desired height.

Align the holes and re-insert the 'e-clips'.

Note: Ensure that all four legs are secured at the same height to prevent rocking and instability.

5. WIDTH ADJUSTMENT (M00870 ONLY)

Remove the 'e-clip' and slide the crossbar to the desired position.

Align the holes and re-insert the 'e-clip'.

Note: Ensure that the e-clip is fully inserted before using.

6. OPTIONAL FLOOR FIXING KIT (M24308)


The floor fixing kit provides a permanent fixing solution for the economy toilet frames. It helps to prevent the toilet frame from moving out of position or becoming unstable during use.

6.1 Fitting Legs to Product:

Remove the original leg extensions by removing the 'e-clips', as shown in section 4.

Insert the four leg extensions and position at the desired height. Align the holes and insert the 'e-clip'.

6.2 Fitting Legs to Floor:

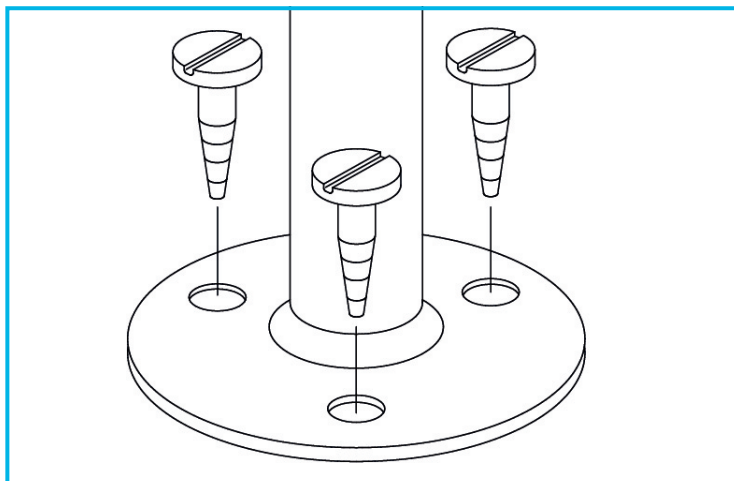

Choose a fixing method which is suitable for the construction of the floor. A total of 12 screws must be used, 3 per leg.

Note: Adjust the toilet frame to the desired width (M00870 only) before securing to the floor.

7. MAINTENANCE

Check that all fixing clips are inserted correctly and that there is no excessive movement.

If a floor fixing kit has been fitted, check the fixing screws are secure.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

8. CLEANING INSTRUCTIONS.

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use/storage.

9. ACCESSORIES & SPARE PARTS

M24308 Economy Range Floor Fixing Kit

For more details please visit our website:

www.healthcarepro.co.uk or call Customer Services.

10. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

cycling Centres.

11. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk