

4200BE - Able Assist Adjustable Sling

Prior to Use:

Inspect the sling for wear and damage, with particular attention being paid to the body of the sling, and stitch lines. Test by pulling on each handle to ensure they are firmly attached and no wear is evident. Understand Loler and your responsibilities as a Carer/Clinician.

If following inspection, there is any doubt of its suitability or safety, then the sling should be placed aside and not used. Then report to your appropriate Line Manager/Organisation. Read the fitting instructions and fully understand the fitting instructions.

Fitting the Belt:

The Able Assist transfer belt has 3 keyhole locks which fit on to the sling location anchors, each side of the handle bar of the Able Assist Transfer Aid. There are three keyhole locks for different sized patients.

Small - red Yellow - medium Green - large

Fit one keyhole lock to the sling location anchor, ensuring that the lock is secure (See Image 'B' below)

Place the sling around the back of the User and secure the keyhole lock in place on the other side of the frame, using the same colour coded keyhole lock.



A - Model 4200BE Able Assist Sling



B - Keyhole lock fitted securely on location anchor