

M00789

Economy Wall Mounted Shower Seat

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

This Shower seat is intended to be used in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering. It can be folded up against the wall when not in use.



Figure 1: Shower Seat (in use)

2. SAFETY

Warning: The maximum user weight must not be more than 136Kg (21.5st) based on the product being fitted to a suitable surface using appropriate fixings by a competent person.

It is recommended that this product should only be installed by a suitably qualified, competent person.

Only install the product in an area where the mounting surface is structurally sound (strengthening modifications to the fixing surface may be required).

DO NOT install in fibreglass/acrylic shower enclosures with insufficient stiffness and strength.

This product must only be secured to the wall using fasteners that are appropriate to the structural base material.

Inspect the product at regular intervals for wear damage and instability, do not use if you suspect a fault.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

Note: Fixings are not supplied with this product.

4. ASSEMBLY/INSTALLATION

Note: You must make sure that the shower base is level where the seat will be installed.

1. Unfold the seat as shown in Figure 1, then adjust the legs to the correct height for the user.
2. Using a spirit level, check the seat is fully level.
3. Mark the fixing positions for the wall mounting brackets on the wall.
4. Using appropriate, stainless steel screws and fixings, secure the brackets to the wall.
5. Check that seat can be raised and lowered without interference, and that the feet are securely in contact with the shower base when lowered.
6. Make sure the seat is fully secure before use.

5. IN USE

Note: Always raise and lower the shower seat using the seat only, not the legs.

1. Pull the seat down from the wall, making sure that both legs are level and in contact with the shower floor.
2. Check the seat it is fully secure before sitting down.
3. Sit centrally on the shower seat.
4. Raise the seat when not in use.

6. MAINTENANCE

1. Inspect the product on a regular basis.
2. Check stability and rigidity.
3. Inspect for wear and damage.
4. Check wall fixings are secure and show no signs of rusting.
5. Do not use if unstable or the rubber feet are damaged.

7. CLEANING INSTRUCTIONS.

Clean regularly, using a general purpose, neutral detergent and warm water.

Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE


NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

The Economy Wall Mounted Shower Seat is part of a range of daily living aids available from NRS Healthcare. To view the full range please visit our website below.

NRS Products you may find useful from our range of bathroom accessories:

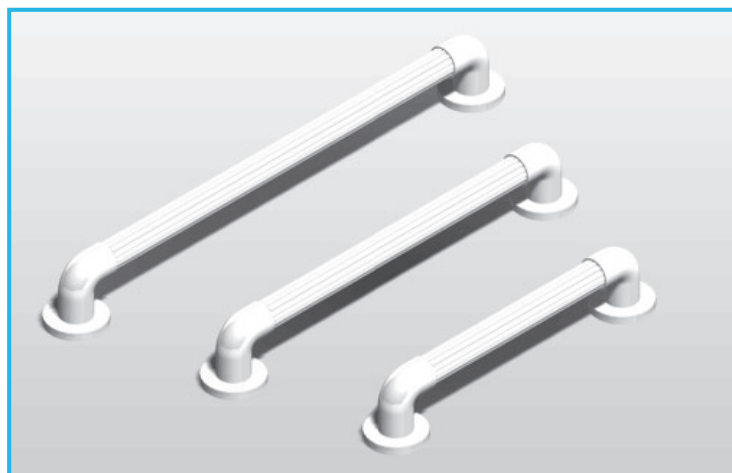

Fluted Grab Rail Range:

F19467 - 300mm White

F19479 - 450mm White

F19480 - 600mm White

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk