

User Instructions

Corner Shower Stool

H84059

Maximum User Weight



General Safety Warning

Please ensure these instructions are fully read and understood before use. Retain in a safe place for future reference.

Intended Use

The Corner Shower Stool is designed to sit in the corner of a shower, taking up minimal space, and is height adjustable so that it can be set at the correct height to assist standing.

Unpacking:

The stool is supplied fully assembled. Please check for any obvious damage before use.

Fitting Instructions:

Check the shower tray can take the recommended weight before use.

Adjust the legs using the "E" clips to the required height for comfortable standing.

When adjusting the legs, ensure there is no gap between the "E" clip and the frame.

Ensure that all the legs are fixed in the same position to prevent rocking.

Periodically check the legs for excess movement and rubber feet and seat for damage. Do not use if damaged.

Safety Instructions:

Take particular care when leaning forward to clean lower extremities (use a long handled sponge to reduce bending if required).

Do not stand on the stool.



Cleaning Instructions

Clean regularly, use a general purpose, mild detergent and warm water.

Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

To disinfect, wipe with a mild chloride disinfectant (solution of 1000 parts per million). Autoclave at 80 °C. Ensure fully dried after cleaning.

Important:

Do not place weight on the seat when removed from the frame. It is designed to be supported by the frame, and only in this position will it conform to the maximum weight limit stated.

Manufactured in the UK for **NRS Healthcare**
Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill, Coalville,
Leicestershire, LE67 1UB, UK

Tel. Main Office: 0330 100 00 98

Tel. Customer Services: 0845 120 4522

Email: customerservice@nrs-uk.co.uk