

User Instructions

M00778 - Folding Shower Chair

**Maximum
User
Weight**



General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Intended Use

The Folding Shower Chair is intended to be used in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering. It can be folded for storage when not in use.

Specification

Max User Weight: 120 Kg (18¾st)

Assembly and Height Adjustment

- 1 - Open the frame so that the arms are pointing forwards, 90 degrees to the back rest.
- 2 - Raise the seat frame and fit into the locations on the arm sections. Ensure they are correctly located. Test with your hands to ensure the chair is stable and secure.
- 3 - Adjust to the correct height by removing the pin clips, sliding the leg extensions up/down the frame tube to the appropriate height and replace the pin clip.
- 4 - Ensure all four leg extensions are secured to the same adjustment hole to ensure even height.
- 5 - To fold; raise the seat frame from the locations and lower. Turn one arm section into the seat frame and the other rotate behind the back rest.

Using the Shower Chair

When fully open, secure, and adjusted to the correct height, place in shower. Ensure all four legs are level and in contact with the shower floor. Test lightly with hands before sitting. Sit centrally on the shower stool.



Safety Precautions

Do not exceed the maximum user weight limit. Inspect at regular intervals for wear and damage. Do not use if rubber feet are damaged.

Cleaning Instructions

Clean regularly, wiping down with mild detergent /disinfectant solution.

The **Folding Shower Chair** is part of a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range please visit our website www.nrs-uk.co.uk or request a catalogue from the number below.

IMPORTANT

For information on how to obtain the User Instructions in other languages and formats for use by people with visual, reading or cognitive impairments, please contact:

info@nrs-uk.co.uk or telephone 0845 120 4522. Additional copies of this user instruction are available on our website: www.nrs-uk.co.uk

NRS Healthcare

Sherwood House, Cartwright Way
Forest Business Park, Bardon Hill
Coalville, Leicestershire. LE67 1UB UK

Tel: 0845 120 4522 Fax: 0845 121 8112
Email: customerservice@nrs-uk.co.uk

www.nrs-uk.co.uk