

Product Technical Data Sheet 033 Plastic Fluted Grab Rail Range

Product Range

F19467	Plastic Fluted Grab Rail—300mm (12")
F19479	Plastic Fluted Grab Rail—450mm (18")
F19480	Plastic Fluted Grab Rail—600mm (24")

Product Description & Intended Purpose

Recommended for internal use to provide a safer environment for those who feel unsteady on their feet and in need of support.

Product Features

- Warm to touch polypropylene plastic.
- Fluted design aids comfort and grip.
- White to complement existing décor, appliances and furniture.
- Circular flange clips over the fixing points to keep product discreet and easy to clean.



Technical Specification

Dimensions (mm)

- Rail Diameter: 36
- Rail to Wall Clearance: 50

Maximum Load: 95Kg (15st)

Installation Information: *(Note: for detailed information refer to User Instructions)*

- The product should be installed by a qualified and competent person.
- Consider the location of the rail so as to be in an optimum position for the user and tasks they wish to complete.
- Check the structure of the wall it is to be fixed to, avoid electrical wiring or plumbing.
- Secure rail in position, strengthening modifications to the fixing surface may be required.
- Perform a final tighten and inspection of the product once installation is complete.
- Vigorously pull on the rail to test the installation. The rail should be rigid and stable.

PTDS033

Plastic Fluted Grab Rail Range

Safety Precautions

- Only install the product in an area where the mounting surface is structurally sound.
- Only install the product using fixings which are compatible with both the rail and fixing surface.
- Do not exceed the maximum user load.
- Do not use the rail to tie objects to or hang items from.
- This product is designed for a single installation.
- Do not place matches or cigarettes on or near this product.

Cleaning

- Clean regularly, use a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- After cleaning always rinse with water and wipe thoroughly with a damp cloth to remove any cleaning material residue.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water
- Allow the product to fully dry before use.

Care & Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the fixing plate and the fit against the fixing surface to ensure there is no excessive movement.
- Any suspected fault or damage should immediately be reported to the supplier and the product taken out of action.

Accessories / Related Products

None

FAQ's

- Are the wall fixings supplied with this rail?

No, the type of fixings required will depend upon the type of wall that you fix it to.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK