

USER INSTRUCTIONS

PLASTIC TRANSFER BOARD

M10966

Plastic Transfer Board

General Safety Warning

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

To assist users with limited mobility to transfer between small gaps (less than 150 mm) either assisted or by self transfer. The surfaces should be of similar height, for example; wheelchair to bed/chair.

2. SPECIFICATION

Material: High density polyethylene with non-slip pads on underside.

Dimensions: 495 x 200 x 10 mm

Weight: 950 g (2 lb)

3. SAFETY

Warning: The maximum user weight must not be more than 127 kg (20 st).

Remove from packaging, check product surfaces are undamaged.

If you see any damage or suspect a fault do not use the product, contact your supplier immediately.

Do not use on excessive slopes.

Inspect at regular intervals, for signs of wear or damage.

4. IN USE

1. Place the transfer surfaces next to each other at right angles, and as close as possible.
2. Remove any detachable arms.
3. Position the Transfer Board to bridge the narrow gap between the two surfaces (less than 150 mm) making sure that the board is adequately and equally supported on each surface.
4. The end-user is now able to shuffle across the board, using arms and legs where possible. Care must be taken to ensure that fingers do not become trapped under the Transfer Board.
5. Where possible, the end-user should push down with their hands during transfer in order to lift themselves slightly, thus avoiding rubbing on the board. This is particularly important where there is a loss of sensation.
6. Assistance from a carer may be required in cases of severe limitation of mobility.

5. CLEANING INSTRUCTIONS.

Wipe all surfaces periodically with a cloth moistened with a neutral detergent and hot water, or with a dilute disinfectant solution.

Do not immerse the board in liquid, or use abrasive cleaners.

Regular inspection of the board is recommended for signs of damage to the surfaces, which may cause increased friction and damage to the skin.

6. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

7. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk