



Instruction manual


W-FBCT Wireless Transmitter



UK Version

The **Alerta Wireless Transmitter** is for use with Alerta Wireless Chair Alertamat, Wireless Bed Alertamat and Wireless Floor+ Alertamat. It uses radio frequency technology to connect to Alerta Wall Point Receiver with a wireless range of up to 95 meters. Once paired with Alerta Wall Point Receiver it will only transmit to that Wall Point Receiver and will not interfere with other Alerta Wall Point Receivers in use nearby.


Set-up Instructions

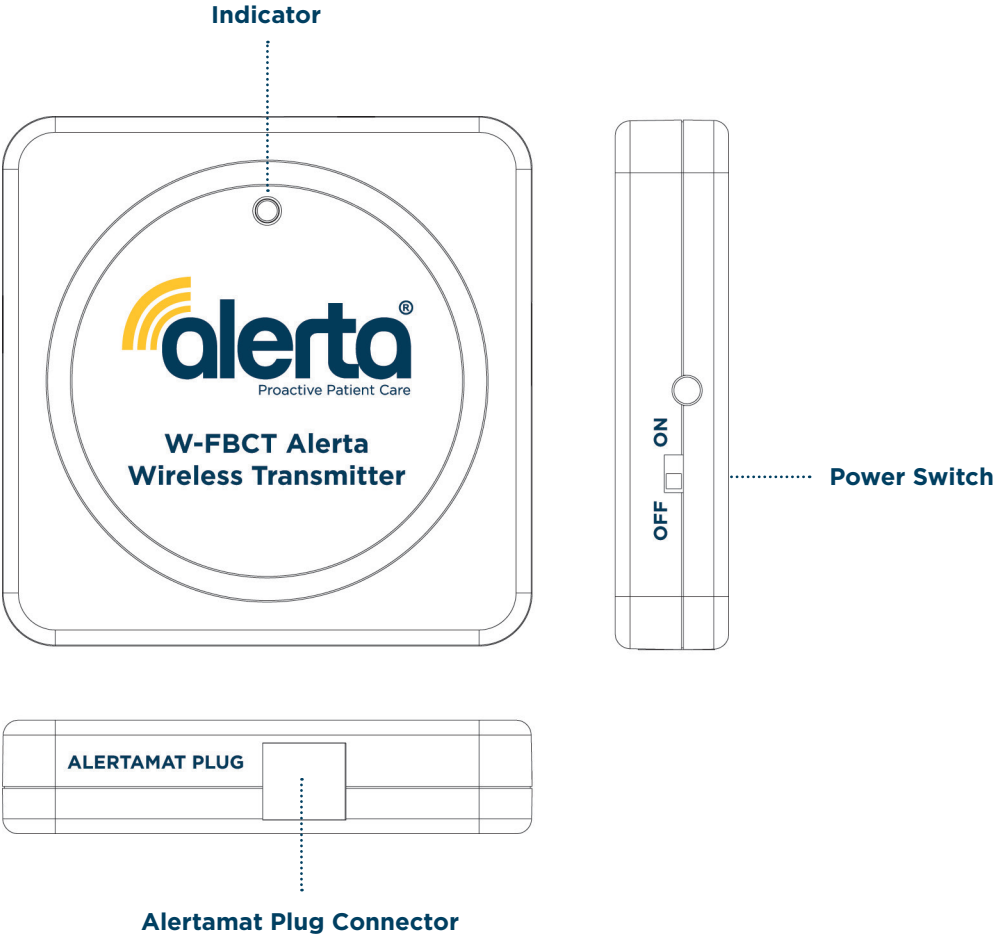

Set-up Instructions

- 1. Remove rear case and install 2 x AAA batteries with the positive (+) and negative (-) ends facing in the proper direction. Then replace rear cover.
 - 2. Turn on power at power switch, the indicator blinks once to show that the power is normal. If the indicator keeps flashing red, it indicates that the battery level is low then replace battery instantly.
 - 3. Have Alertamat (Alerta Wireless Chair Alertamat, Wireless Bed Alertamat or Wireless Floor+ Alertamat) to hand and plug the Alertamat wire to the Alertamat Plug Connector in the transmitter. Place Alerta Wireless Transmitter in the pocket of the Alertamat and seal the pocket closed. Test the Alertamat before use.
- Remember to switch Alerta Wireless Transmitter to 'OFF' if not in use for a period of time.

CAUTION



Improper installation of the battery or the installation of the incorrect battery type can result in damage to the contacts and cause improper operation.

Making **cutting-edge** medical equipment **affordable**

Head Office

Alerta Medical
4 Symington Place
Riverside Business Park
Irvine
KA11 5DE
United Kingdom

www.alertamedical.com

