

SELECTING THE AUDIO AND VIBRATING ALARM

There are three alarm modes to select from: beeps (#), vibration(§§) and beep+vibration(§§#). To select different alarm mode, slide the switch to BEEP, VIB or BEEP+VIB. In BEEP+VIB the alarm sound the beeps for 2 seconds followed by vibration for 2 seconds, repeating.

RESETTING THE FIVE ALARMS

All the 5 alarms automatically reset to their original alarm times at 0:13 daily.

FILLING THE PILL BOX

Fill the numbered compartments (1 to 5) with pills by rotating the cover to access each compartment.

TURNING ON BACKLIGHT

Press LIGHT to turn on the backlight for 5 seconds.

RESETTING THE UNIT

When the pillbox shows abnormal function, such as "frozen display", "abnormal display", "frozen button" etc, you need to initialize the unit by pressing RESET using a pointed object such as paper clip. The RESET button is located in the inside of the top body.

www.tabtime.com