




## 3 in 1 Rail

- Standard
- Splayed
- Fold-down

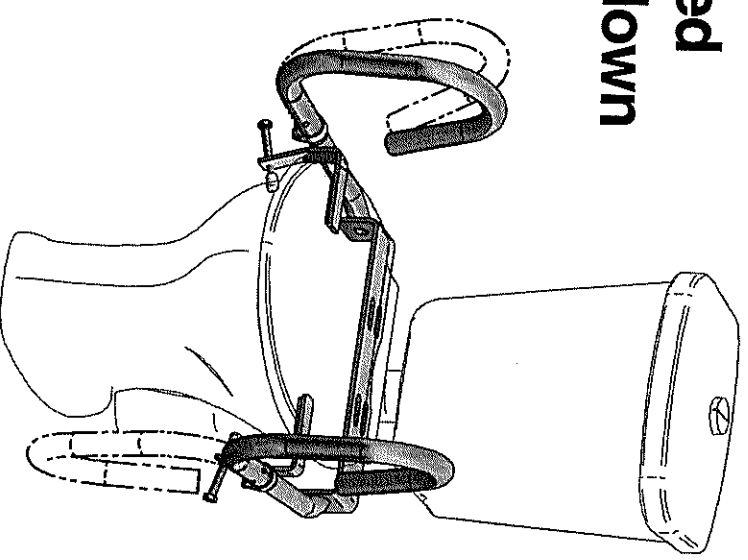

**Disclaimer:** Throne Accessories Pty Limited will not be held responsible for any injury or damage resulting from the product being incorrectly installed (including, but not limited to, placement on unstable or ill-founded toilet bowls) or for inappropriate use. These rails have been tested up to 110 kilograms and approved by a NATA registered laboratory applying AS3696:part 8(8.4)

[www.throne.com.au](http://www.throne.com.au)

# Combination Rail positions

Standard Rail

## To move Handles

- Loosen clamps (use Hexkey supplied in box)
- Press finger buttons in
- Rotate Handle to new position
- Buttons will click into place
- Tighten clamp if new position is permanent


## To remove Handles

- Loosen clamps (use Hexkey supplied in box)
- Press finger buttons in
- Pull Handles off mounting tube


Splayed Rail

Fold Down Rail


either or both sides can be folded down

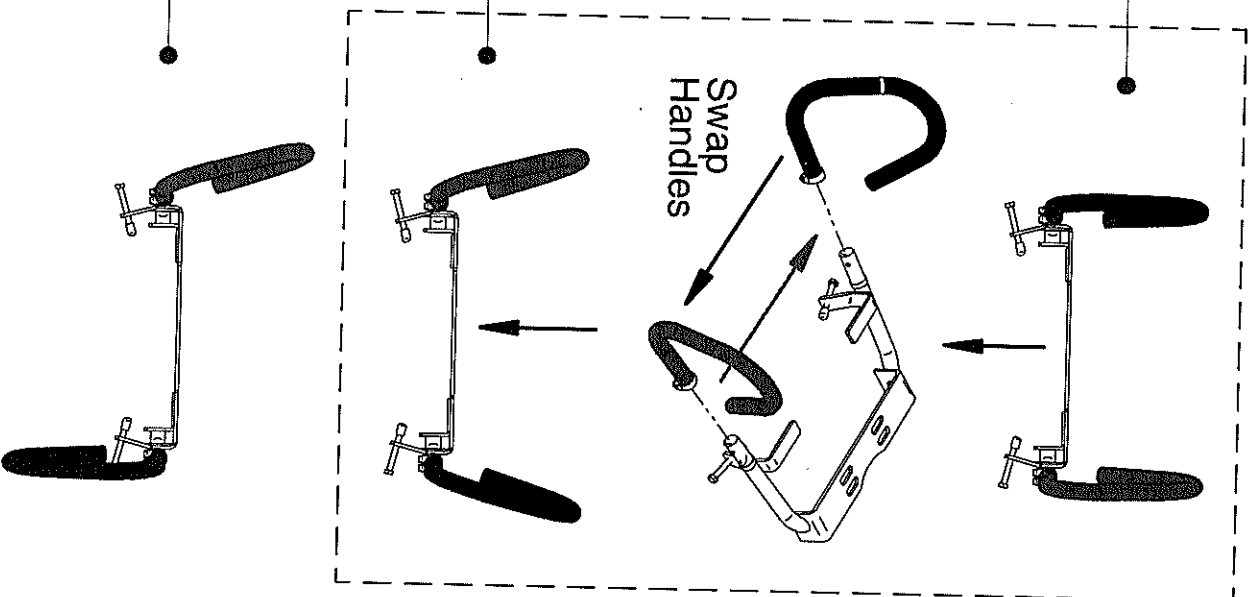

## Box contents

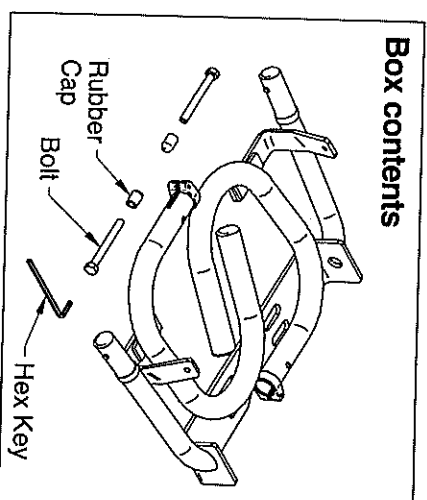

1. Remove toilet seat, install Throne Rail and replace toilet seat using the same screws

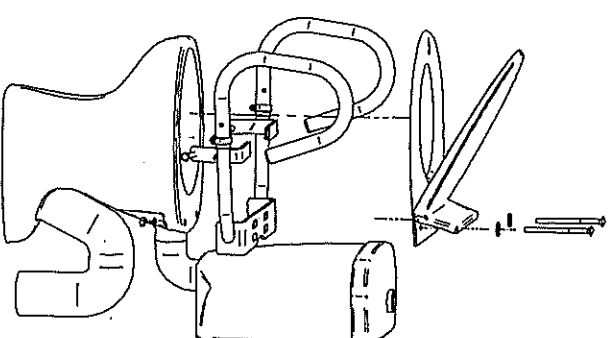

2. Hand-tighten side bolts so that rubber caps firmly contact side of toilet bowl

