

# USER INSTRUCTIONS

## NRS SHOWER STOOL WITH SWIVEL SEAT

M99138

NRS Shower Stool with Swivel Seat

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

#### 1. INTENDED USE

For use in a shower. It has a swivelling seat to enable the user to sit and easily position themselves. The seat can be easily removed for cleaning.



#### 2. SAFETY

**Warning:** The maximum user weight must not be more than 127 kg (20 st).

Floor surfaces must be level and even to avoid rocking.

All four ferrules must be in contact with the floor at all times.

Periodically check all legs for total movement.

Ensure the seat is securely fixed

Do not exceed the maximum user weight limit.

Inspect at regular intervals for wear and damage.

#### 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.


### Maximum User Weight



#### 3.1 Contents:

1. Seat x 1
2. Legs x 4
3. Leg Extensions x 4
4. Plastic Wing Nut x 1

#### 4. ASSEMBLY/INSTALLATION

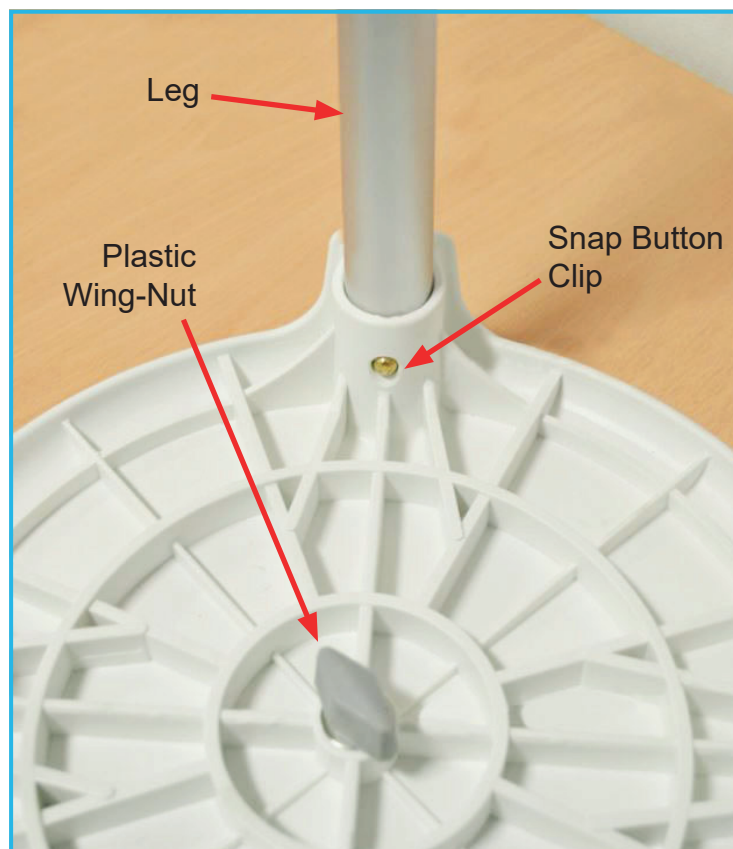

**Figure 1:**

With the seat upturned push the legs into the seat orifices, as shown in Figure 1 making sure the button clips, snap into position.

Check the legs are rigid and do not move.

Fit the Leg Extensions into the legs, again make sure the button clips snap into position.

Adjust the seat to the desired height by sliding the leg extensions, make sure all the extensions are at the same height.

**Note:** Make sure all button clips have fully engaged the holes and the seat is fully secure before use.

## 5. IN USE

Place in shower. Ensure all four legs are level and in contact with the shower floor.

Sit centrally on the shower stool.

## 6. CLEANING INSTRUCTIONS

Clean regularly, wipe with a soft cloth and a mild detergent/disinfectant solution. Do Not use abrasive cleaners.

**Note:** The swivelling part of the seat can be removed for cleaning by removing the plastic wing nut, as shown in Figure 1.

## 7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)