

Product Range

P11255	A-Series 4 Wheel Aluminium Rollator — Red
P11267	A-Series 4 Wheel Aluminium Rollator — Blue
P11279	A-Series 4 Wheel Aluminium Rollator — Silver

Product Description & Intended Purpose

The A-Series Rollator is designed to provide support and assistance to those unable to, or who lack the confidence to, walk without some level of guidance. This rollator is intended for indoor and outdoor use.

Product Features

- Lightweight, aluminium frame, folds for easier transport and storage.
- Under seat bag storage with concealed zip pocket.
- Padded backrest for added comfort.
- Height adjustable, easy grip handles and brake levers.



Technical Specification

Dimensions (mm)

- Handle Height: 830 — 980 (32½" — 38½")
- Width between Handles: 450 (17¾")
- Seat size: 350 X 300 (13¾" - 11¾")
- Seat Height: 580 (22¾")
- Product Folded: 250 X 620 X 990 (10" X 24½" X 39")
- Wheel Diameter: 180 (7")
- Product Weight: 6.5kg (14¼lb)
- Maximum User Weight: 135kg (21st)

Safety Precautions

- **Do not exceed the maximum user weight 21st (135kg).**
- Ensure the rollator is locked in the open position before use.
- Only use as a walking aid in the forward direction.
- Do not use on stairs, escalators or steep hills with gradients greater than 1 in 8.
- Ensure both brakes are locked and that the backrest is fitted before using the seat.
- Do not move the rollator when the seat is occupied.
- Do not sit on the seat when the rollator is on an incline.
- Do not hang items from the handles as this may affect the stability of the rollator.

PTDS084 A-Series 4 Wheel Rollator

Installation Information: *(Note: for detailed information refer to User Instructions)*

- This product is delivered folded and requires assembly before use.
- Assembly time is approximately 5 minutes.
- No specialist tools or skills are required.

Cleaning

- Ensure all dirt and debris is removed from around the wheels, axle and brakes.
- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Care & Maintenance

- Check all fasteners are correctly located and securely fitted.
- Check the wheels rotate freely.
- Ensure the brakes are functioning correctly. When squeezed or locked the two rear wheels should not be able to rotate.
- Check for general wear or damage. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

FAQ's

- Is this suitable for indoor and outdoor use?
Yes, as long as you have the space indoors it can be used for both.
- How do I measure the correct height for the handles?
Stand with your feet flat on the floor, in your usual footwear, with your arms held loosely at your side. Measure the height from the floor to your wrist, this is the optimum height for the handles.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK