

Product Range

N85146 Home Helper Trolley — Flat Pack

Product Description & Intended Purpose

The Home Helper Trolley is ideal for indoor use by those with reduced mobility. The strong and versatile frame is height adjustable and is fitted with two trays to provide a safe means for users to move items around the home.

Product Features

- Powder coated steel frame with two rigid plastic trays.
- Design allows for a natural walking gait.
- Raised lips on trays help prevent spillages.
- Removable trays for easy cleaning.
- Height adjustable to suit a range of users.



Technical Specification

Dimensions (mm)

- Handle Height (mid point): 825 - 950 (32" - 37")
- Overall Width X Depth: 460 x 500 (18" x 19½")
- Tray Size: 310 x 430 (12" x 17")
- Wheel Diameter: 100 (4")
- Tray Capacity: 5kg (11lb)
- Maximum User Weight: 20st (127kg)
- Product Weight: 8kg (1¼st)

Installation Information *(Note: for detailed information refer to User Instructions)*

- This product is supplied flat pack and requires assembly before use.
- No specialist tools or skills are required to assemble this product.
- Assembly time is approximately 5 minutes.

Safety Precautions

- Do not use the trolley to assist when attempting to stand from a seated position.
- Do not attempt to sit on the trolley.
- Do not overload the trolley trays. When possible it is advisable to position items centrally.
- Do not use the trolley outdoors.

PTDS002

N85146 Home Helper Trolley - Flat Pack

Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- If required the trays can be removed for cleaning, simply unclip from the frame. To refit, engage the clips on the front of the tray on the front cross tube and secure by clipping the rear edge of the tray onto the rear cross tube.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- Allow product to dry thoroughly prior to use or storage.

Care & Maintenance

- Regularly check the condition of the castors and remove any build up of dirt or debris.
- Check that the trays are securely fitted and are level prior to use.
- Check that all the fixing clips are inserted correctly and that there is no excessive movement.
- Periodically check the trolley for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of action.

Accessories / Related Products

- F17628— Dycem non-slip mat - Red
- F17639— Dycem non-slip mat—Blue

FAQ's

- Do the wheels (castors) Lock?
No
- Do the wheels (castors) swivel?
Yes the wheels have 360 ° rotation.
- Are the trays removable?
Yes the trays can be removed for cleaning.
- How do I measure the correct height for the handles?
Stand with your feet flat on the floor, in your usual footwear, with your arms held loosely at your sides and measure the height from the floor to your wrist.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville, Leicestershire. LE67 1UB UK