

User Instructions

Step stool & Hand rail

F19121 – Step Stool
L64087 – Step & Hand Rail

Maximum
User Weight



General Safety Warning

Please ensure these instructions are fully read and understood before use. Retain in a safe place for future reference.

Intended Use

This product is available as a simple step stool or with a fixed handrail to offer the user additional support. It is ideal for general household use and especially suitable for helping users enter and exit the bath, or for use in the kitchen.

Specification

Sturdy coated tubular steel frame fitted with non-slip rubber ferrules and a fluted rubber, slip resistant, covered wooden top.

Care and Use Important

Ensure an even floor surface to avoid rocking. Check all fixings are secure, before and after use. **DO NOT EXCEED** the maximum user weight of the product. Check the product monthly for signs of wear and tear. Pay particular attention to the rubber feet.

Cleaning

The Step Top can be cleaned with soap and water and the frame with a non-abrasive cream cleaner or detergent solution.



Manufactured for **NRS Healthcare**
Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill,
Coalville, Leicestershire.
LE67 1UB, UK

Tel. Main Office: 0330 100 00 98
Tel. Customer Services: 0845 120 4522
Email: customerservice@nrs-uk.co.uk

This product is from a range of over 3000 daily living aids available from NRS Healthcare. To view the full range visit www.nrs-uk.co.uk or request a catalogue from the above number.