



**SEERS MEDICAL LTD**  
Kenton Road  
Debenham  
Suffolk  
IP14 6LA  
United Kingdom  
**t:** +44(0) 1728 861590  
**f:** +44(0) 1728 861868

**e:** [info@seersmedical.com](mailto:info@seersmedical.com)  
**w:** [www.seersmedical.com](http://www.seersmedical.com)

## **OPERATING INSTRUCTIONS FOR THE SEERS MEDICAL OPERATOR CHAIR RANGE**

### **Seat Height:**

Higher = without weight - lift lever A and adjust seat height

Lower = with weight on seat - lift lever A and adjust seat height

### **Back Rake:**

To adjust the rake position of the backrest

Lift lever B and adjust the back rest into position

To lock in desired position - gently push down lever B

### **Seat Tilt:**

To adjust the seat tilt lift lever C

To lock in desired position - gently push down lever C

### **Back Rest Height Manual:**


Turn hand-wheel D anti-clockwise to release clamp

Adjust back rest to desired height and turn hand-wheel D clockwise to tighten clamp

### **Backrest Height Ratchet:**

To adjust the backrest height - simply lift the backrest to the desired position

To re-set - lift the backrest to the top of its travel and the backrest will slide back to the bottom ready to be lifted into the desired position

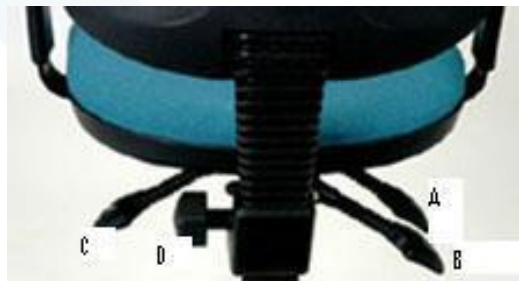

**CONTROL LEVERS A Band C MARKED WITH INSTRUCTION GRAP**