

## Product Range

P05229 Children's Toilet Platform

### Product Description & Intended Purpose

The Children's Toilet Platform is designed to enable children who's feet cannot reach the floor when sat on a standard toilet to be able to climb onto a platform and then sit comfortably with their feet supported.



### Product Features

- Contemporary design with comfortable handles.
- Lightweight steel frame, powder coated white.
- Height adjustable platform.
- Designed to provide optimum stability.
- Soft mat for extra comfort and grip.



### TECHNICAL SPECIFICATION

SAFE WORKING LOAD 76KG (12ST)

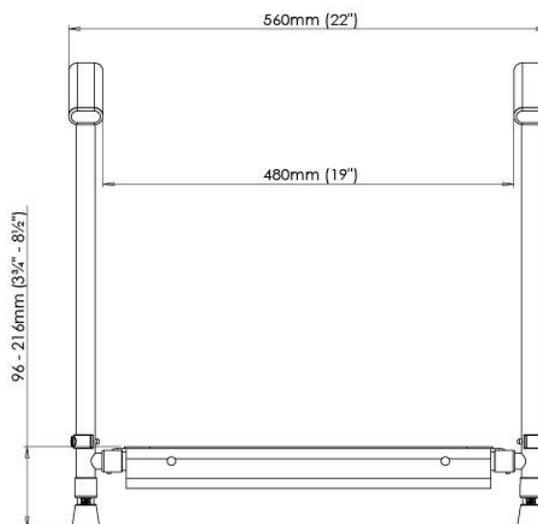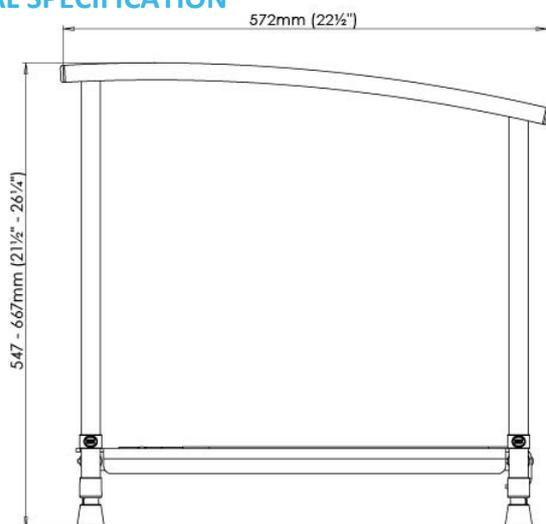

**Installation Information:** *(Note: for detailed information refer to User Instructions)*

#### Assembly:

- Attach the platform to the LH & RH frame by depressing the button clips and sliding the platform tubes over both the LH & RH frame side tubes. Ensure the button clip clicks back into place.
- Insert the leg extensions by sliding into the frame legs to correct height and insert an 'e-clip'.

#### To adjust the Height:

- Remove the 'e-clips' and slide the legs to the desired position and re-insert the 'e-clips'.

## Product Technical Data Sheet 050 P05229 Children's Toilet Platform

### Safety Precautions

- Ensure all four legs are set to the same height to prevent rocking and instability.
- Do not exceed the safe working load of the product, 76 Kg (12 Stone).
- Ensure the toilet platform is pushed up to the toilet as close as possible.

### Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
- Allow product to dry thoroughly prior to use or storage.

### Care & Maintenance

- Check that all fixing clips are inserted correctly and that there is no excessive movement.
- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the mat, ferrules and frame fixings.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

### Accessories / Related Products

None

### FAQ's

- How do I know what the best height is to set the platform to for my child?

*When your child is seated on the toilet, the feet should rest comfortably on the toilet platform with the knees bent at a normal seated angle.*

### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville, Leicestershire. LE67 1UB UK