

# Product Technical Data Sheet 090

## N72255 Children's Shower Chair

### Product Range

N72255 Children's Shower Chair

### Product Description & Intended Purpose

The children's shower chair is intended for use in a bathroom or shower, to provide a stable seated surface for children who are unable to stand for long periods of time.

### Product Features

- Height adjustable legs to suit different height users.
- Plastic moulded seat complete with drainage holes.
- Rubber ferrules for added grip on wet floors.
- Corrosion resistant aluminium frame.



### Technical Specification

**Seat Height:** 237mm - 385mm (9" - 15")      **Maximum User Weight :** 64kg (10st)

**Seat Width:** 380mm (15")

**Seat Depth:** 300mm (12")

**Footprint:** 468mm X 462mm (18.5" X 18")

### Safety Precautions

- Do not exceed the maximum user weight 10st, 64kg.
- Do not attempt to stand on the chair.
- Ensure the seat is correctly and securely fitted to the frame prior to use.
- Check all legs have been adjusted to equal length prior to use.
- Do not use if rubber feet are damaged.

### Installation Information: *(Note: for detailed information refer to User Instructions)*

- Secure the seat to the frame using the four stainless screws provided.
- Push an extension leg into each of the four corners of the frame and push the anti rattle collar up around the frame.
- Adjust the leg to the required height by sliding in and out of the frame tube.
- Align the holes and insert an 'e-clip' to secure in position. Repeat for all four legs.

## PTDS090 N72255 Children's Shower Chair

### Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

### Care & Maintenance

- Check that all screws and fixing clips are securely fitted.
- Regularly check the condition of the rubber ferrules.
- Periodically inspect the shower chair for signs of general wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

### Accessories / Related Products

N/A

### FAQ's

### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK