

USER INSTRUCTIONS

FOLDING SHOWER CHAIR

M00778

Folding Shower Chair

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

For use in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering. It can be folded for storage when not in use.



ASSEMBLY/INSTALLATION

3.1 Unfolding/Folding the Frame:



2. SAFETY

Warning: The maximum user weight must not be more than 120 kg (18 ¾ st).

Ensure all four legs are level and in contact with the shower floor.

Test lightly with hands before sitting. Sit centrally on the shower stool.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

Open the frame so that the arms are pointing forwards, 90 degrees to the back rest.

Raise the seat frame and fit into the locations on the arm sections. Ensure they are correctly located.

Test with your hands to ensure the chair is stable and secure.

To fold; raise the seat frame from the locations and lower. Turn one arm section into the seat frame and the other rotate behind the back rest.

3.2 Height Adjustment:

Adjust to the correct height by removing the pin clips, sliding the leg extensions up/down the frame tube to the appropriate height and replace the pin clip.

Ensure all four leg extensions are secured to the same adjustment hole to ensure even height.

4. MAINTENANCE

Inspect at regular intervals for wear and damage.

5. CLEANING INSTRUCTIONS

Clean regularly, wiping down with mild detergent / disinfectant solution.

Allow product to fully dry prior to use/storage.

6. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

7. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk