

User Instructions

NUVO™ Toilet Frame Range

Nuvo™ Toilet Frames	Free Standing	Floor Fixed	Max User Weight
Nuvo Standard	L70270	L70282	30st /190kg
Nuvo Extra Wide	L70294	L70300	40st /250kg
Nuvo Width Adjustable	L70312		30st /190kg
Floor Fix Kit for above		L70324	
Nuvo Children's	L98346		10st / 64kg
Children's Floor Fix kit		M09423	



General Safety Warning

Please ensure these instructions are fully read and understood before installation/use. Retain these instructions in a safe place for future reference.

Intended Purpose

The Nuvo Toilet Frame range is designed to assist those who have difficulty in lowering onto, or rising from a standard toilet. They can also be used in conjunction with a raised toilet seat.

Fitting Instructions

1. Remove from the packaging and check the frame is free from damage.
2. The frame is delivered ready assembled.
3. To use the frame, first adjust to the desired height, remove the plastic E clips connecting the legs and leg extensions. Slide the leg extensions to the required position ensuring that all legs are equal in height. Replace the plastic clips ensuring that each clip is fully connected.

Width Adjustable Models

1. Adjust frame to desired height, as above.
2. Adjust width by removing 'E' clips, sliding horizontal bars to required width, aligning holes and replacing 'E' clips.

Floor Fixing

1. Adjust frame to desired height, as above.
2. A total of 12 screws must be used to fix the frame to the floor, using fixing methods appropriate to the floor's construction.

Safety Precautions

- Do not exceed the maximum user weight limit.
- Ensure an even floor surface to prevent Rocking.
- Inspect at regular intervals for wear and damage.
- Do not use if rubber feet are damaged.

Cleaning Instructions

- Clean frame and armrests regularly, use a general purpose, neutral detergent and warm water.
- Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.
- To disinfect, wipe with a mild chloride disinfectant (solution of 1000 parts per million).

This product is from a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range please visit our website or contact us, Tel: 0345 121 8111

Manufactured in the UK for

NRS Healthcare,
Sherwood House,
Cartwright Way,
Forest Business Park,
Bardon Hill,
Coalville,
Leicestershire,
LE67 1UB,
United Kingdom.

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

www.nrshealthcare.co.uk

