

# User Instructions

## P01560 Height Adjustable Shower Chair

### General Safety Warning

**Please ensure these instructions are fully read and understood before installation/use. Retain these instructions in a safe place for future reference.**

### 1.0 Intended Use

This shower chair is intended to be used in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering.

### 2.0 Safety Precautions

- Do not exceed the maximum user weight.
- Do not use if the feet are damaged.
- Ensure all legs are secured at the same height to keep the chair stable.
- Do not attempt to move the chair when occupied.

### 3.0 Unpacking

Remove chair from packaging. Check all parts are present and there is no visible damage. If you see or suspect a fault do not use and contact your supplier immediately.

### 4.0 Height Adjustment

Product is supplied fully assembled.

To adjust height:

- Remove 'e-clip' and slide leg extension to required height.
- Align the holes and re-insert the 'e-clip'.
- Repeat for all four legs ensuring they are set to equal height.
- Check the seat is level and the product is stable before use.



**Maximum  
User Weight:**



### 5.0 Cleaning Instructions

- Clean regularly, use a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
- Do not autoclave.
- Dry the seat thoroughly before use.

## 6.0 Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.
- Check that all fixing clips are inserted correctly and there is no excessive movement in the legs or seat.
- If you see or suspect a fault do not use and contact your supplier immediately.

## 7.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**From the NRS Healthcare Range, you may also like:**

### **Nuvo Adjustable Bath Step**

Code No: M11077



### **Spa Curved Grab Rail**

M53527 Length: 355mm (14")  
M53539 Length: 480mm (19")  
M85553 Length: 620mm (24 1/2")



### **Combi Grabber**

N72358 Length: 660mm (26")  
N72369 Length: 813mm (32")



## **NRS Healthcare**

Sherwood House, Cartwright Way,  
Forest Business Park, Bardon Hill,  
Coalville, Leicestershire.

LE67 1UB UK

Tel. Main Office: 0330 100 00 98

Tel. Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)



**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**